## AC Adapter HHP-CM01 Instruction Manual

**Issue Date:**      **Date de publication :**      **Ausgabedatum:**  
**Data di pubblicazione:**      **Fecha de publicación:**      **Utgiftfedatum:**      **2022-12-10**  
**Дата выпуска:**      **Teslim Tarihi:**      **تاريخ الإصدار:**

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**Intended Use**  
This device is an AC Adapter intended for use with applicable Omron devices. The Omron devices which are compatible has this model mentioned on the rating label. In case compatibility is not clear please consult an authorized Omron service provider.

- Please read this instruction manual and applicable Omron device manual thoroughly before using the unit.  
Please keep for future reference.

#### Important safety information

**▲ Warning:** Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- Do not use the AC adapter if the device or the power cord is damaged. Turn off the power and unplug the power cord immediately.
- Plug the AC adapter into the appropriate voltage outlet. Do not use in a multi-outlet plug.
- Never plug in or unplug the power cord from the electric outlet with wet hands.
- Do not disassemble or attempt to repair the AC adapter.

**▲ Caution:** Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

- Fully insert the power plug into the outlet.
- When disconnecting the power plug from the outlet, be sure to safely pull from the power plug.
- Do not pull from the power cord.
- When handling the power cord, take care not to do the following:
  - Do not damage.
  - Do not break it.
  - Do not tamper with it.
  - Do not forcibly bend or pull.
  - Do not twist.
  - Do not bundle during use.
  - Do not pinch.
  - Do not place under heavy objects.
- Wipe any dust off of the power plug.
- Do not use gasoline, thinners or similar solvents to clean the device.
- Disconnect the power plug before cleaning.
- AC adapter cable may cause accidental strangulation in infants.
- Use this AC adapter only for compatible main Omron devices. Use of unsupported adapters may damage and/or may be hazardous to the device. While using this device, make sure that no mobile phone or any other electrical devices that emit electromagnetic fields is within 30cm. This may result in degradation of performance of the device.

#### Using the AC Adapter

Note: Make sure to use an easily accessible power socket in which to connect and disconnect the AC adapter.

- Insert the AC adapter plug into the AC adapter jack of the device.
- Plug the AC adapter into an electrical outlet.

To disconnect the AC adapter, unplug the AC adapter from the electrical outlet first, and then remove the AC adapter plug from the device.

#### Maintenance and Storage

To protect your device from damage, please follow the directions below:

- Store the device in a clean, safe location.
- Do not use any abrasive or volatile cleaners.
- Do not wash the device or immerse them in water.
- Do not use gasoline, thinners or similar solvents to clean the device.
- Disconnect the device from the electrical outlet and use a soft dry cloth to clean.
- Changes or modification not approved by the manufacturer will void the user warranty.
- Do not disassemble or attempt to repair the device or components.

<b>EN</b> Product Description	<b>AC Adapter</b>
<b>EN</b> Model	<b>HHP-CM01</b>
<b>EN</b> Electrical Rating	<b>AC100-240 V, 50-60 Hz, 0.12-0.065 A</b>
<b>EN</b> Output voltage	<b>6V DC / 700 mA</b>
<b>EN</b> Durable period (Service life)	<b>5 years</b>
<b>EN</b> Operation conditions:	
<b>EN</b> Ambient temperature	<b>10 to 40 °C</b>
<b>EN</b> Relative humidity	<b>15 to 90 % (non-condensing)</b>
<b>EN</b> Ambient Pressure	<b>700-1060 hPa</b>
<b>EN</b> Transportation and storage conditions:	
<b>EN</b> Ambient temperature	<b>-20 to 60 °C</b>
<b>EN</b> Relative humidity	<b>10 to 90 % (non-condensing)</b>
<b>EN</b> Weight	<b>47.5 g</b>
<b>EN</b> Dimensions (box)	<b>54 (w) × 72 (h) × 96 (l) mm</b>
<b>EN</b> IP Classification*	<b>IP21</b>
<b>EN</b> Protection against electric shock	<b>Class II ME equipment</b>
<b>EN</b> Contents	<b>AC Adapter, Instruction Manual</b>
<b>EN</b> Notes:	
<b>EN</b> * These specifications are subject to change without notice.	
<b>EN</b> * IP classification is degrees of protection provided by enclosures in accordance with IEC 60529. The AC adapter is protected against solid foreign objects of 12.5 mm diameter and greater such	