- ⓔⓃ Instruction Manual
- ⓔⓣ Istruzioni per l'uso
- ⓔⓔ Gebrauchsanweisung
- ⓔⓕ Mode d'emploi
- ⓔⓗ Gebruiksaanwijzing
- ⓔⓖ Manual de instrucciones
- ⓔⓗ Руководство по эксплуатации
- ⓔⓗ دليل التعليمات

**Body Composition Monitor**  
**BF214 (HBF-214-EBW)**  
بي أف 214

**Misuratore della composizione corporea**  
**Körperanalyse-Monitor**  
**Moniteur de composition corporelle**  
**Lichaamscompositiemeter**  
**Monitor de Composición Corporal**  
**Монитор состава тела OMRON BF214 (HBF-214-EBW)**  
جهاز مراقبة الدهون بالجسم

**EN** Thank you for purchasing this body composition monitor. It's intended to measure and display the following body composition parameters: body weight, body fat (in %), skeletal muscle (in %) and Body Mass Index (BMI). The 4-sensor technology provides clinically validated body measurements.

This unit is intended to be operated by adults who can understand this instruction manual.

 Please read this instruction manual carefully before use and for further information on the individual functions. Please keep this Instruction Manual at hand all the time for future reference.

 Please read all of the information in "12 Notes of safety" carefully.

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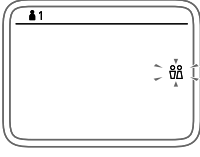
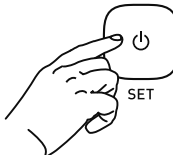
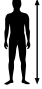
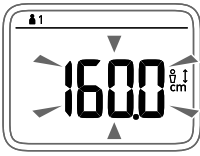
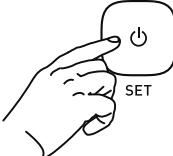
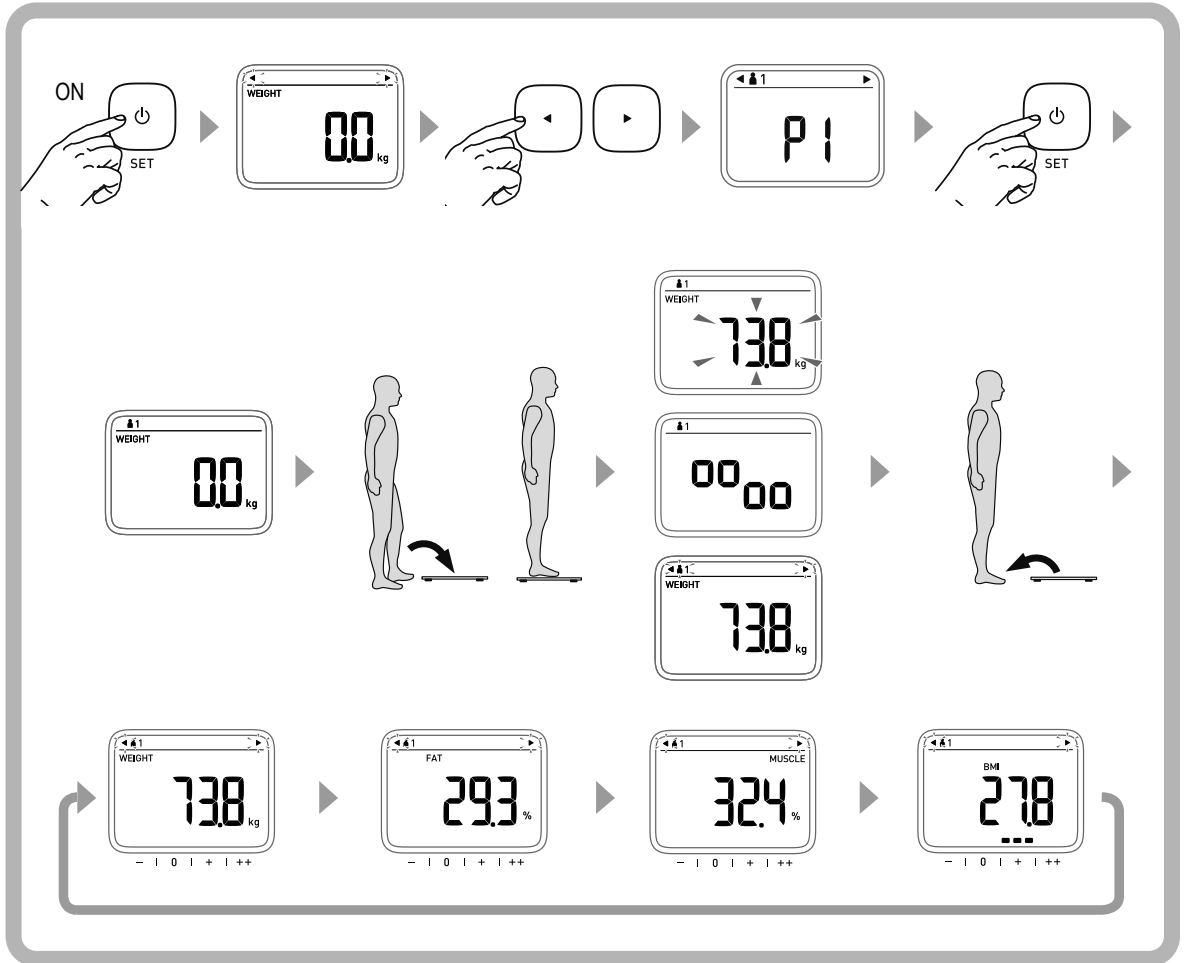
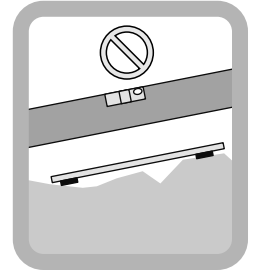
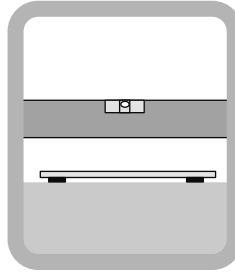
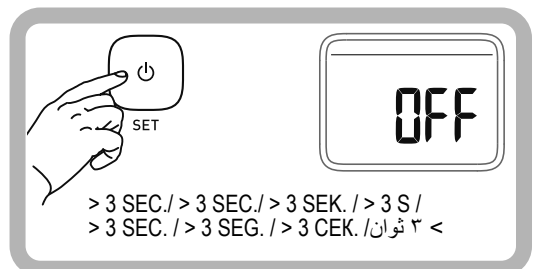
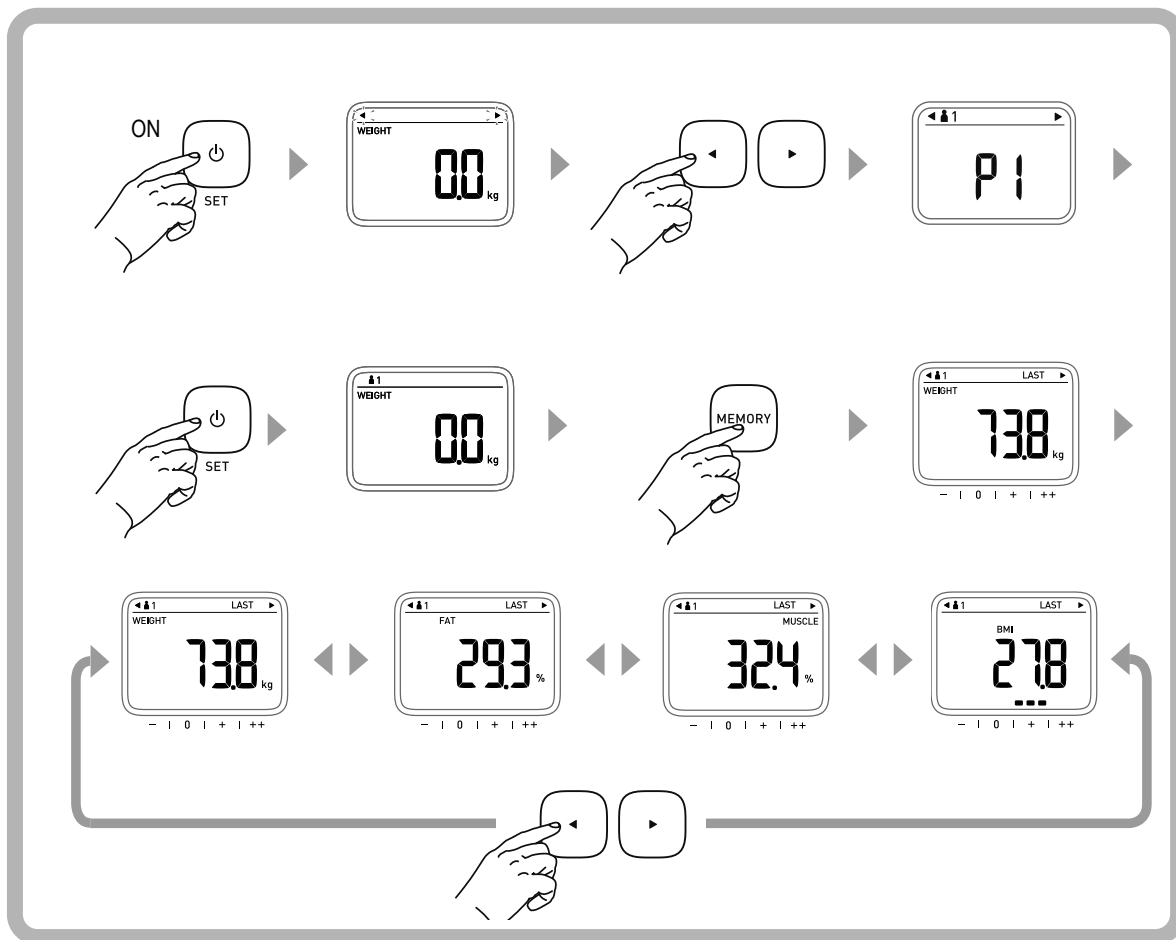
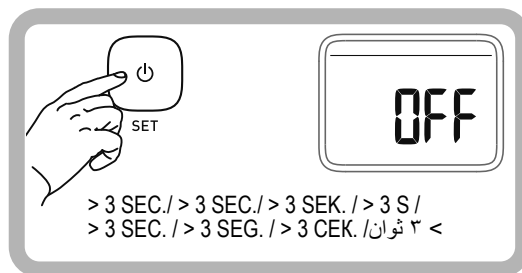
	AGE ETÀ ALTER ÂGE LEEFTIJD EDAD ВОЗРАСТ الفئة العمرية	low basso niedrig bas laag bajo низкое منخفض	normal normale normal normal normaal normal нормальное طبيعي	high alto hoch haut hoog alto высокое ارتفاع	very high molto alto sehr hoch très haut zeer hoog muy alto очень высокое عالية جدا
	10	< 16.1%	16.1 - 32.2%	32.3 - 35.2%	≥ 35.3%
	11	< 16.3%	16.3 - 33.1%	33.2 - 36.0%	≥ 36.1%
	12	< 16.4%	16.4 - 33.5%	33.6 - 36.3%	≥ 36.4%
	13	< 16.4%	16.4 - 33.8%	33.9 - 36.5%	≥ 36.6%
	14	< 16.3%	16.3 - 34.0%	34.1 - 36.7%	≥ 36.8%
	15	< 16.1%	16.1 - 34.2%	34.3 - 36.9%	≥ 37.0%
	16	< 15.8%	15.8 - 34.5%	34.6 - 37.1%	≥ 37.2%
	17	< 15.4%	15.4 - 34.7%	34.8 - 37.3%	≥ 37.4%
	18 - 39	< 21.0%	21.0 - 32.9%	33.0 - 38.9%	≥ 39.0%
40 - 59	< 23.0%	23.0 - 33.9%	34.0 - 39.9%	≥ 40.0%	
60 - 80	< 24.0%	24.0 - 35.9%	36.0 - 41.9%	≥ 42.0%	
	10	< 12.8%	12.8 - 27.9%	28.0 - 31.8%	≥ 31.9%
	11	< 12.6%	12.6 - 28.5%	28.6 - 32.6%	≥ 32.7%
	12	< 12.3%	12.3 - 28.2%	28.3 - 32.4%	≥ 32.5%
	13	< 11.6%	11.6 - 27.5%	27.6 - 31.3%	≥ 31.4%
	14	< 11.1%	11.1 - 26.4%	26.5 - 30.0%	≥ 30.1%
	15	< 10.8%	10.8 - 25.4%	25.5 - 28.7%	≥ 28.8%
	16	< 10.4%	10.4 - 24.7%	24.8 - 27.7%	≥ 27.8%
	17	< 10.1%	10.1 - 24.2%	24.3 - 26.8%	≥ 26.9%
	18 - 39	< 8.0%	8.0 - 19.9%	20.0 - 24.9%	≥ 25.0%
40 - 59	< 11.0%	11.0 - 21.9%	22.0 - 27.9%	≥ 28.0%	
60 - 80	< 13.0%	13.0 - 24.9%	25.0 - 29.9%	≥ 30.0%	

HD McCarthy et al, in the International Journal of Obesity, Vol. 30, 2006, and by Gallagher et al., American Journal of Clinical Nutrition, Vol. 72, Sept. 2000, and classified into four levels by Omron Healthcare.

Skeletal muscle / Massa muscolare scheletrica / Skelettmuskeln / Muscle squelettique / Skeletspiermassa / Músculo esquelético /  
Скелетная мускулатура / العضلات الهيكلية

	AGE ETÀ ALTER ÂGE LEEFTIJD EDAD ВОЗРАСТ الفئة العمرية	low basso niedrig bas laag bajo низкая منخفض	normal normale normal normal normaal normal нормальная طبيعي	high alto hoch haut hoog alto высокая ارتفاع	very high molto alto sehr hoch très haut zeer hoog muy alto очень высокая عالية جدا
	18 - 39	< 24.3%	24.3 - 30.3%	30.4 - 35.3%	≥ 35.4%
	40 - 59	< 24.1%	24.1 - 30.1%	30.2 - 35.1%	≥ 35.2%
	60 - 80	< 23.9%	23.9 - 29.9%	30.0 - 34.9%	≥ 35.0%
	18 - 39	< 33.3%	33.3 - 39.3%	39.4 - 44.0%	≥ 44.1%
	40 - 59	< 33.1%	33.1 - 39.1%	39.2 - 43.8%	≥ 43.9%
	60 - 80	< 32.9%	32.9 - 38.9%	39.0 - 43.6%	≥ 43.7%

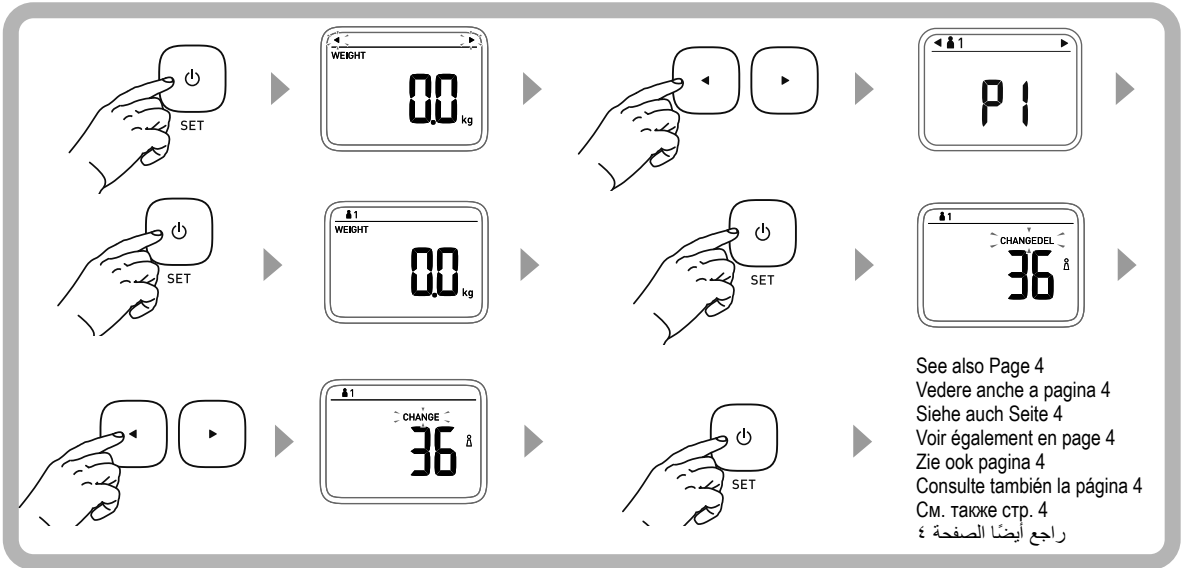
According to OMRON HEALTHCARE figures

BMI (Body Mass Index / Indice di massa corporea / Körpermassenindex / Indice de masse corporelle / Body Mass Index / Índice de masa corporal / Индекс массы тела / مؤشر كتلة الجسم

BMI	Classifications (by the WHO) / Classificazioni (dell'OMS) / Classifications (par l'OMC) / Classificaties (door de WHO) / Clasificaciones (según la OMS) / Классификации (по данным ВОЗ) / التصنيفات (من قبل منظمة الصحة العالمية)	
< 18.5	Underweight / Sottopeso / Untergewicht / Poids insuffisant / Ondergewicht / Peso inferior al normal / Недостаточная масса тела / وزن ناقص	-
18.5 - 24.9	Normal / Normale / Normal / Normal / Normaal gewicht / Normal / Нормально / طبيعي	0
25 - 29.9	Overweight / Sovrappeso / Übergewicht / Excédent de poids / Overgewicht / Sobrepeso / Тучность / وزن زائد	+
≥ 30	Obese / Obeso / Adipös / Obésité / Obesitas / Obesidad / Ожирение / بدانة	++

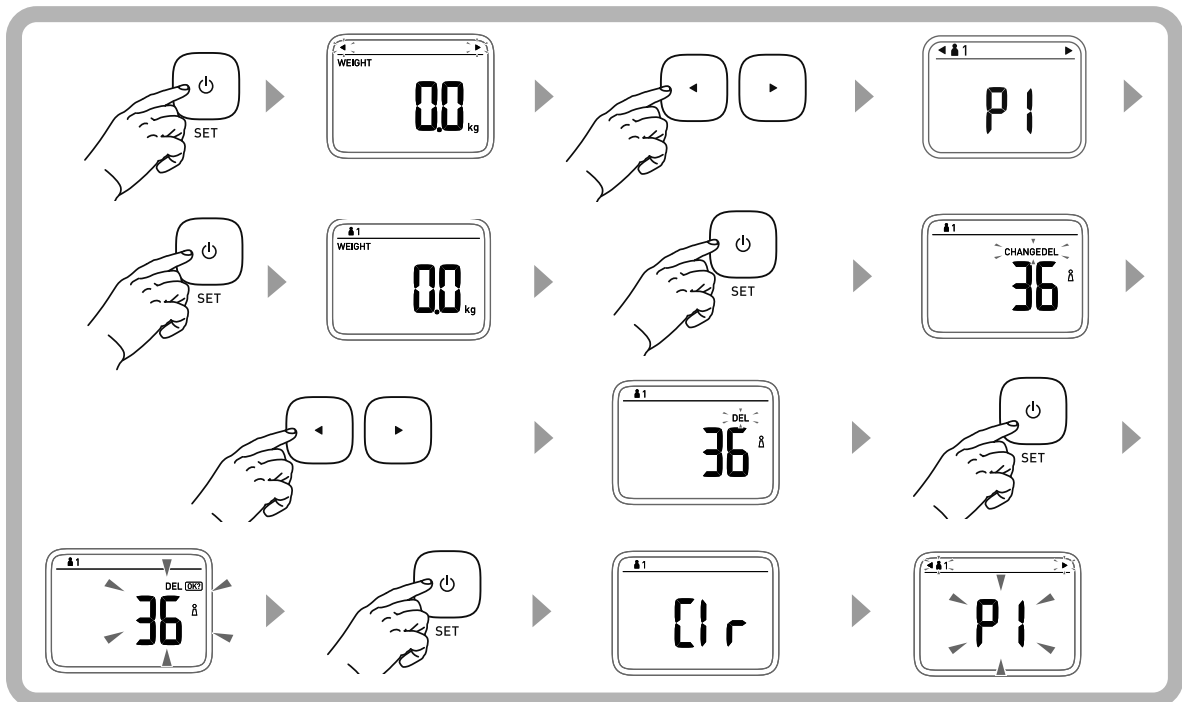
- (EN) Changing personal data
- (IT) Modifica dei dati personali
- (DE) Persönliche Daten ändern
- (FR) Modification des données personnelles

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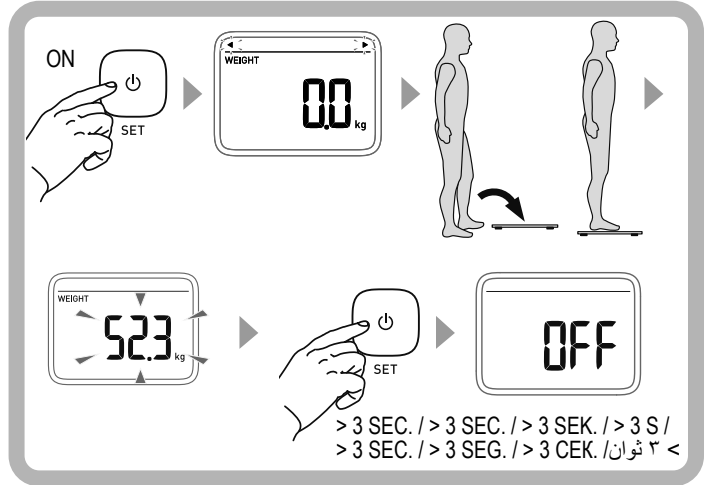
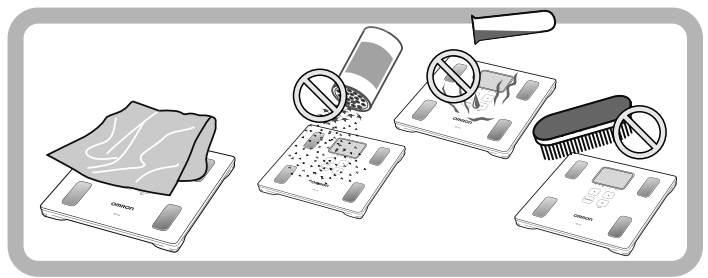
\*Wipe the unit with a soft dry cloth. Do not wipe the unit with benzene or paint thinner. If necessary, use a cloth moistened with water or detergent and squeeze it well before wiping the unit, then wipe dry with a dry cloth.

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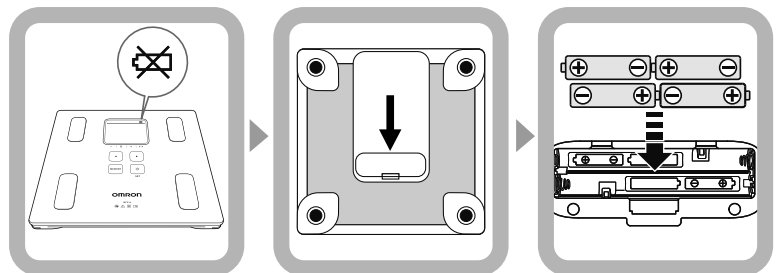
## EN Principle of body composition calculation

### Body fat has low electric conductivity

The BF214 measures the body fat percentage by the Bioelectrical Impedance (BI) method. Muscles, blood vessels and bones are body tissues with a high water content that conducts electricity easily. Body fat is tissue that has little electric conductivity. The BF214 sends an extremely weak electrical current of 50 kHz and less than 500  $\mu\text{A}$  through your body to determine the amount of fat tissue. This weak electrical current is not felt while operating the BF214.

In order for the scale to determine body composition, it uses the electrical impedance, along with your height, weight, age and gender information to generate results based on OMRON's data of body composition.

### Recommended measurement times

During the course of a day, the amount of water in the body tends to gradually shift to the lower limbs. This is why there is a tendency for the legs and ankles to swell in the evening or at night. The ratio of water in the upper body and lower body is different in the morning and evening, and this means that the electrical impedance of the body also varies. Being aware of the times when the body fat percentages shift within your own daily schedule will assist you in obtaining an accurate trending of your body fat. It is recommended to use this unit in the same environment and daily circumstances. We recommend to do the measurement always in the morning after getting up with an empty bladder.

Avoid taking measurements under the following conditions:

- Immediately after vigorous exercise, after a bath or sauna.
- After drinking alcohol or a large amount of water, after a meal (about 2 hours)

If a measurement is taken under these physical conditions, the calculated body composition may differ significantly from the actual one because the water content in the body is changing.

### What is BMI (Body Mass Index)?

BMI uses the following simple formula to indicate the ratio between weight and height of a person.  
 $BMI = \text{weight (kg)} / \text{height (m)} / \text{height (m)}$  or  $BMI = \text{weight (lb)} / \text{height (inches)} / \text{height (inches)} \times 703$   
 The OMRON BF214 uses the height information stored in your personal profile number or when information in the Guest Mode to calculate your BMI classification.

### What is Body Fat Percentage?

Body fat percentage refers to the amount of body fat mass in regards to the total body weight expressed as a percentage.

Body fat percentage (%) =  $(\text{Body fat mass (kg)} / \text{Body weight (kg)}) \times 100$

### What is Skeletal Muscle?

Skeletal muscle can be increased through exercise and other activity. Increasing the ratio of skeletal muscle means that body can burn energy more easily, which means that it is less likely to turn to fat, and makes it easier to lead an energetic lifestyle.

### The reason calculated results may differ from actual body composition.

The body composition measured by this unit may significantly differ from the actual body composition in the following situations:

- Elderly people (over 81 years old) / People with a fever / Body builders or highly trained athletes / Patients undergoing dialysis / Patients with osteoporosis who have very low bone density / Pregnant women / People with swelling
- These differences may be related to changing ratios of body fluid and/or body composition.

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## Notes of safety

Please read this instruction manual carefully before use and for further information on the individual functions.

### Danger:

- Never use this unit in combination with the following medical electronic devices.
  - Medical electronic implants such as pacemakers.
  - Electronic life support systems such as an artificial heart/lung.
  - Portable medical devices such as electrocardiograph.
- This unit may cause the above-mentioned medical electronic devices to malfunction, posing a considerable health risk to users of these devices.

- Do not disassemble, repair, or remodel the unit.
- Do not use batteries not specified for this unit. Do not insert the batteries with the polarities in the wrong direction.
- Replace worn batteries with new ones immediately.
- Remove the batteries from this unit when you are not going to use it for a long period of time (approximately three months or more).
- Do not use batteries of a different kind together.
- Do not use new and worn batteries together.
- Keep this unit out of the reach of young children. Contains small parts that may cause a choking hazard if swallowed by infants.

### Warning:

- Never start weight reduction or exercise therapy solely based on your own judgement. Be sure to follow the instructions of a doctor or specialist.
- Do not use the unit on tiles or other surfaces that may be slippery, such as a wet floor.
- Do not subject the unit to strong shocks, such as vibrating or dropping the unit on the floor.
- Do not jump or bounce on the unit.
- Do not use this unit after taking a bath, or when your body, hands, or feet are wet.
- Do not step on the edge of the unit.
- Do not let a physically handicapped person use this unit without any accompanied assistance. Use a handrail or so when stepping on the unit.
- When the device malfunctions, it may get hot. Areas with a possible rise in temperature:
  - Around the battery compartment area: 105°C (max.)
  - The electrodes and operation buttons: 48°C (max.)
- If the device malfunctions and does not operate, stop using it immediately. Do not apply the electrodes or keep touching the operation buttons for 10 minutes or more.

### General Advice:

- Do not place this unit on cushioned floor surface such as on a carpet or a mat. Correct measurement may not be possible.
- Do not step on the operation buttons. Doing so may cause malfunction.
- Do not place this unit in highly humid environment, where water may splash, under direct sunshine, in a place where the air conditioner blows directly, or near fire.
- Do not use this unit for purposes other than measuring body weight and body fat percentage.
- Disposal of this product and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products.
- Always keep the unit clean before use.
- Clean the scale before using the unit with people who have a skin or foot disease.
- Do not wash the unit with water.
- Do not wipe the unit with benzene or paint thinner.
- When cleaning the unit, take care not to touch the buttons and change or delete the personal data.
- If storing the unit in a vertical position, always secure it so that it will not fall over.
- Do not store the unit in the following conditions:
  - Where water may get in.
  - Extreme high temperature and humidity, direct sunshine, and dusty places.
  - Where there will be sudden shock or vibration.
  - In storage places of chemicals or where corrosive gas is present.
- Do not place objects on the unit or store it upside-down.
- Use the battery within recommended period mentioned to it.

### Caution:

- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities.
- This unit does not support the standards required for professional use.
- Stand on the unit bare-footed.
- During measurement, make sure that no mobile phone or any other electrical devices that emit electromagnetic fields is within 30cm of this device. This may result in incorrect operation of the device and/or cause an inaccurate reading.

### Troubleshooting

In case of any of the below problems occur during measurement, first check that no other electrical device is within 30cm. If the problem persists, please refer to the table below.

Error Display	Possible cause	Correction
Err1	You step off the unit during the measurement.	Do not step off the unit till the measurement is completed.
	Your feet are not correctly positioned on the electrodes.	Make sure that you are standing on the unit correctly and try again.
Err2	The measurement position is not stable or your feet are not placed correctly.	Remain still and do not move during measurement.
	Feet are too dry.	Slightly moisten the soles of your feet with a damp towel and try again.
Err5	The unit is not set up correctly.	Remove the batteries. Wait one minute. Reinstall the batteries. Touch the ON-OFF-SET button and turn on the power. Retake the measurement.
Err	You step onto the unit before 0.0 kg (0.0 lb) is displayed.	Wait until 0.0 kg (0.0 lb) is displayed before stepping onto the unit.
	The unit moved before 0.0 kg (0.0 lb) is displayed.	Do not move the unit until 0.0 kg (0.0 lb) is displayed
	Movement during measurement.	Do not move till measurement is completed.
	Your weight is over 150.0 kg (330.0 lb or 23 st 8.0 lb).	This unit can only be used by people weighing less than 150.0 kg (330.0 lb or 23 st 8.0 lb).

### Technical Data

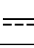
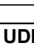
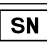
Product Category	Body Composition Analyzers
Product Description	Body Composition Monitor
Model (Code)	BF214 (HBF-214-EBW)
Display	Body Weight: 2 to 150 kg with an increment of 0.1 kg / 4.4 to 330.0 lb with an increment of 0.2 lb / 4.4 lb to 23 st 8.0 lb with an increment of 0.2 lb Body Fat percentage: 5.0 to 60.0% with an increment of 0.1% Skeletal Muscle percentage: 5.0 to 50.0% with an increment of 0.1% BMI: 7.0 to 90.0 with an increment of 0.1 BMI classification: - (Underweight) / 0 (Normal) / + (Overweight) / ++ (Obese) • The age range for the BMI, BMI classification and Body Fat percentage is 10 to 80 years. • The age range for the Skeletal Muscle percentage is 18 to 80 years.
Memory:	Last (previous) measurement results
Set Items Personal data information:	The following information for up to 4 people can be stored. Height: 100.0 to 199.5 cm, 3'4" to 6'6 3/4" (increment of 1/4") Age: 10 to 80 years old Gender: Male/Female
Weight Accuracy	2.0 kg to 40.0 kg: ± 0.4 kg, 40.0 kg to 150.0 kg: ± 1% / 4.4 lb to 88.2 lb: ± 0.88 lb, 88.2 lb to 330.0 lb: ± 1% / 4.4 lb to 6 st 4.2 lb: ± 0.88 lb, 6 st 4.2 lb to 23 st 8.0 lb: ± 1%
Accuracy (S.E.E.)	Body Fat percentage: 3.5%, Skeletal Muscle percentage: 3.5%
Durable Period	5 years
Power Supply	4 AAA batteries (R03) (You may also use AAA alkaline batteries (LR03).)
Battery Life	Approximately one year (When AAA manganese batteries are used with four measurements a day at a room temperature of 23°C.)
Operating Temperature / Humidity / Air Pressure	+10 to 40°C / 30 to 85% RH (non-condensing) / 700 to 1060 hPa
Storage and Transport Temperature / Humidity / Air Pressure	-20 to 60°C / 10 to 95% RH (non-condensing) / 700 to 1060 hPa
Protection Against Electric Shock	Internally powered ME equipment
Applied Part	Type BF (Foot Electrodes)
IP Classification	IP21
*IP classification is degrees of protection provided by IEC 60529.	
This unit is protected against solid foreign objects of diameter 12.5mm such as a finger and greater. This unit is protected against the ingress of vertically falling water drops.	
Weight	Approximately 1.6 kg (including batteries)
External Dimensions	Approximately 285 (W) × 28 (H) × 280 (D) mm
Package Contents	Body composition monitor, 4 AAA manganese batteries (R03), instruction manual

Note: Subject to technical modification without prior notice.

Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

OMRON HEALTHCARE EUROPE B.V. guarantees this product for 3 years after date of purchase.

The guarantee does not cover battery, packaging and/or damages of any kind due to misuse (such as dropping or physical misuse) caused by the user. Claimed products will only be replaced when returned together with the original invoice / cash ticket. This product is calibrated at the time of manufacture. If at any time you question the accuracy of measurements, please contact your authorized OMRON distributor. In general it is recommended to have the device inspected every 2 years to ensure correct functioning and accuracy.

Description of symbols that, depending on a model, can be found on the product itself, product sales package or IM			
	Applied part - Type BF Degree of protection against electric shock (leakage current)		Temperature limitation
IP XX	Ingress protection degree provided by IEC 60529		Humidity limitation
	CE Marking		Atmospheric pressure limitation
	Symbol of Eurasian Conformity		Direct current
	Need for the user to consult the instructions for use		This product should not be used by persons with medical implants, e.g. heart pacemakers, artificial heart, lung or other electronic life support systems.
	Medical Device		Date of Manufacture
	UKCA Marking		Unique device identifier
	Serial number		

### Important information regarding Electromagnetic Compatibility (EMC)

This device manufactured by OMRON HEALTHCARE Co., Ltd. conforms to EN60601-1-2:2015 Electromagnetic Compatibility (EMC) standard.

Further documentation in accordance with this EMC standard is available at OMRON HEALTHCARE EUROPE at the address mentioned in this instruction manual or at [www.omron-healthcare.com](http://www.omron-healthcare.com)

### Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

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