

Automatic Upper Arm Blood Pressure Monitor M2 Basic (HEM-7121J-E) X2 Basic (HEM-7121J-EO)

Read Instruction manual ① and ② before use.

FR Lire le mode d'emploi ① et ② avant l'utilisation.

DE Lesen Sie vor der Verwendung die Gebrauchsanweisung ① und ②.

IT Leggere il manuale di istruzioni ① e ② prima dell'uso.

ES Lea el manual de instrucciones ① y ② antes del uso.

RU Прочтите руководство по эксплуатации ① и ② перед использованием.

TR Kullanmadan önce, kullanım kılavuzu ① ve ②'yi okuyun.

PL Przed rozpoczęciem korzystania z ciśnieniomierza należy przeczytać instrukcję obsługi ① i ②.

PT Leia o Manual de instruções ① e ② antes de utilizar.

AR اقرأ دليل الإرشادات ① و ② قبل الاستخدام.

EN
FR
DE
IT
ES
RU
TR
PL
PT
AR
Symbols

Overview

FR Présentation générale

DE Überblick

IT Presentazione del prodotto

ES Descripción general


RU Описание прибора

TR Genel Bakış

PL Przegląd

PT Descrição geral

نظرة عامة **AR**

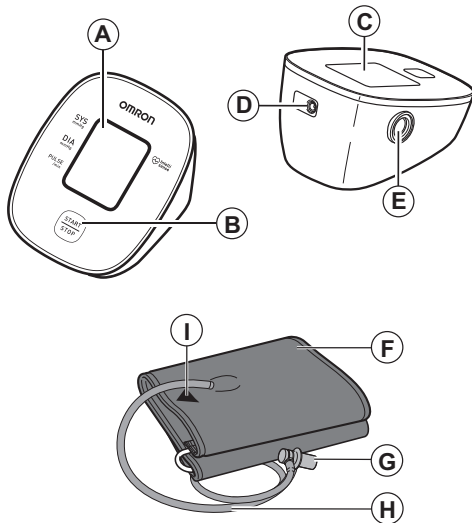

A	<p>Display</p> <p>FR Affichage DE Display IT Display ES Pantalla</p> <p>RU Дисплей TR Ekran PL Wyświetlacz PT Visor</p> <p>الشاشة AR</p>
B	<p>[START/STOP] button</p> <p>FR Bouton [START/STOP] DE [START/STOP]-Taste IT Pulsante [START/STOP] ES Botón [START/STOP]</p> <p>RU Кнопка [START/STOP] TR [START/STOP] düğmesi PL Przycisk [START/STOP] PT Botão [START/STOP]</p> <p>زر [START/STOP] (فاقي/البنب) AR</p>
C	<p>Battery compartment</p> <p>FR Compartiment des piles DE Batteriefach IT Alloggiamento batterie ES Compartimento de las pilas</p> <p>RU Отсек для элементов питания TR Pil bölümü PL Komora baterii PT Compartimento das pilhas</p> <p>حجرة البطاريات AR</p>
D	<p>AC adapter jack</p> <p>FR Prise pour l'adaptateur secteur DE Netzteilanschluss IT Presa per alimentatore CA ES Toma del adaptador de CA</p> <p>RU Гнездо адаптера переменного тока TR AC adaptörü prizi PL Gniazdko zasilacza PT Entrada do adaptador de CA</p> <p>مقيس مهلي التيار المتردد AR</p>

E	Air jack	FR Prise à air	RU Воздушное гнездо
	DE Luftschauchbuchse	TR Hava jakı	
	IT Presa per il tubo dell'aria	PL Przyłącze powietrza	
	ES Toma de aire	PT Tomada de ar	
			مقيس خرطوم الهواء AR
F	Arm cuff	FR Brassard	RU Манжета
	DE Manschette	TR Kolluk	компрессионная
	IT Bracciale	PL Mankiet	
	ES Manguito	PT Braçadeira	
			الشريط الضاغط للزراع AR
G	Air plug	FR Prise de gonflage	RU Воздушный штекер
	DE Luftschauchstecker	TR Hava tapası	
	IT Attacco del tubo dell'aria	PL Wtyczka przewodu powietrza	
	ES Conector para tubo de aire	PT Ficha de ar	
			قابس خرطوم الهواء AR
H	Air tube	FR Tuyau à air	RU Воздуховодная
	DE Luftschauch	TR Hava borusu	трубка
	IT Tubo dell'aria	PL Przewód powietrza	
	ES Tubo de aire	PT Tubo de ar	
			خرطوم الهواء AR

I	Marker	FR Marque	RU Маркер
	DE Markierung	TR İşaretçi	
	IT Contrassegno	PL Znacznik	
	ES Marcador	PT Marcador	
			علامة التحديد AR

1. Introduction

Thank you for purchasing the OMRON Automatic Upper Arm Blood Pressure Monitor. This blood pressure monitor uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

1.1 Safety Instructions

This instruction manual provides you with important information about the OMRON Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all of these instructions. **If you do not understand these instructions or have any questions, contact your OMRON retail outlet or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.**

1.2 Intended Use

This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population. The device detects the appearance of irregular heartbeats during measurement and indicates this via a symbol with readings. It is mainly designed for general household use.

1.3 Receiving and Inspection

Remove this monitor from the packaging and inspect for damage. If this monitor is damaged, DO NOT USE and consult with your OMRON retail outlet or distributor.

2. Important Safety Information

Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety.

Keep for future reference. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.



2.1 Warning

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.
- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.
- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.
- To help avoid strangulation, keep the air tube and AC adapter cable away from infants, toddlers and children.
- This product contains small parts that may cause a choking hazard if swallowed by infants, toddlers and children.

AC Adapter (optional accessory) Handling and Usage

- DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged. If this monitor or the cable is damaged, turn off the power and unplug the AC adapter immediately.
- Plug the AC adapter into the appropriate voltage outlet. DO NOT use in a multi-outlet plug.
- NEVER plug in or unplug the AC adapter from the electric outlet with wet hands.
- DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage


- Keep batteries out of the reach of infants, toddlers and children.



2.2 Caution

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property.

- Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.
- Consult with your physician before using this monitor on an arm where intravascular access or therapy, or an arteriovenous (A-V) shunt, is present because of temporary interference to blood flow and could result in injury.
- Consult with your physician before using this monitor if you have had a mastectomy or lymph node clearance.
- Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising.
- DO NOT take measurements more often than necessary because bruising, due to blood flow interference, may occur.
- ONLY inflate the arm cuff when it is applied on your upper arm.
- Remove the arm cuff if it does not start deflating during a measurement.
- DO NOT use this monitor for any purpose other than measuring blood pressure.
- During measurement, make sure that no mobile device or any other electrical device that emit electromagnetic fields is within 30 cm of this monitor. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.
- DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor.
- DO NOT use this monitor in a moving vehicle such as in a car or on an aircraft.
- DO NOT drop or subject this monitor to strong shocks or vibrations.
- DO NOT use this monitor in places with high or low humidity or high or low temperatures. Refer to section 6.
- During measurement, observe the arm to ensure that the monitor is not causing prolonged impairment to blood circulation.

- DO NOT use this monitor in high-use environments such as medical clinics or physician offices.
- DO NOT use this monitor with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation and/or cause an inaccurate reading.
- Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for at least 30 minutes before taking a measurement.
- Rest for at least 5 minutes before taking a measurement.
- Remove tight-fitting or thick clothing from your arm while taking a measurement.
- Remain still and DO NOT talk while taking a measurement.
- ONLY use the arm cuff on persons whose arm circumference is within the specified range of the cuff.
- Ensure that this monitor has acclimated to room temperature before taking a measurement. Taking a measurement after an extreme temperature change could lead to an inaccurate reading. OMRON recommends waiting for approximately 2 hours for the monitor to warm up or cool down when the monitor is used in an environment within the temperature specified as operating conditions after it is stored either at the maximum or at the minimum storage temperature. For additional information on operating and storage/transport temperature, refer to section 6.
- DO NOT use this monitor after the durable period has ended. Refer to section 6.
- DO NOT crease the arm cuff or the air tube excessively.
- DO NOT fold or kink the air tube while taking a measurement. This may cause an injury by interrupting blood flow.
- To unplug the air plug, pull on the plastic air plug at the base of the tube, not the tube itself.
- ONLY use the AC adapter, arm cuff, batteries and accessories specified for this monitor. Use of unsupported AC adapters, arm cuffs and batteries may damage and/or may be hazardous to this monitor.
- ONLY use the approved arm cuff for this monitor. Use of other arm cuffs may result in incorrect readings.
- Inflating to a higher pressure than necessary may result in bruising of the arm where the cuff is applied. NOTE: refer to "If your systolic pressure is more than 210 mmHg" in section 6 of instruction manual  for additional information.
- Read and follow the "Correct Disposal of This Product" in section 7 when disposing of the device and any used accessories or optional parts.

AC Adapter (optional accessory) Handling and Usage

- Fully insert the AC adapter into the outlet.
- When unplugging the AC adapter from the outlet, be sure to safely pull from the AC adapter. DO NOT pull from the AC adapter cable.
- When handling the AC adapter cable:
 - Do not damage it. / Do not break it. / Do not tamper with it.
 - DO NOT pinch it. / Do not forcibly bend or pull it. / Do not twist it.
 - DO NOT use it if it is gathered in a bundle.
 - DO NOT place it under heavy objects.
- Wipe any dust off of the AC adapter.
- Unplug the AC adapter when not in use.
- Unplug the AC adapter before cleaning this monitor.

Battery Handling and Usage

- DO NOT insert batteries with their polarities incorrectly aligned.
- ONLY use 4 "AA" alkaline or manganese batteries with this monitor. DO NOT use other types of batteries. DO NOT use new and used batteries together. DO NOT use different brands of batteries together.
- Remove batteries if this monitor will not be used for a long period of time.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
- If battery fluid should get on your skin, wash your skin immediately with plenty of clean, lukewarm water. If irritation, injury or pain persists, consult with your physician.
- DO NOT use batteries after their expiration date.
- Periodically check batteries to ensure they are in good working condition.

2.3 General Precautions

- When you take a measurement on the right arm, the air tube should be at the side of your elbow. Be careful not to rest your arm on the air tube.



- Blood pressure may differ between the right and left arm, and may result in a different measurement value. Always use the same arm for measurements. If the values between both arms differ substantially, check with your physician on which arm to use for your measurements.
- When using an optional AC adapter, make sure not to place your monitor in a location where it is difficult to plug and unplug the AC adapter.

Battery Handling and Usage









- Disposal of used batteries should be carried out in accordance with local regulations.
- The supplied batteries may have a shorter life span than new batteries.




Remember to have a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure.

Please use the Blood Pressure Diary to keep records of several readings over a certain period of time. To download PDF files of the diary, visit www.omron-healthcare.com.

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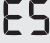3. Error Messages and Troubleshooting

If any of the below problems occur during measurement, check to make sure that no other electrical device is within 30 cm. If the problem persists, please refer to the table below.

Display/Problem	Possible Cause	Solution
 E1 appears or the arm cuff does not inflate.	The [START/STOP] button was pressed while the arm cuff is not applied.	Press the [START/STOP] button again to turn the monitor off. After inserting the air plug securely and applying the arm cuff correctly, press the [START/STOP] button.
	Air plug is not completely plugged into the monitor.	Insert the air plug securely.
	The arm cuff is not applied correctly.	Apply the arm cuff correctly, then take another measurement. Refer to section 4 of instruction manual (2).
	Air is leaking from the arm cuff.	Replace the arm cuff to the new one. Refer to section 9 of instruction manual (2).
 E2 appears or a measurement cannot be completed after the arm cuff inflates.	You move or talk during a measurement and the arm cuff does not inflate sufficiently.	Remain still and do not talk during a measurement. If "E2" appears repeatedly, inflate the arm cuff manually until the systolic pressure is 30 to 40 mmHg above your previous readings. Refer to section 6 of instruction manual (2).
	Due to the systolic pressure is above 210 mmHg, a measurement cannot be taken.	
 E3 appears	The arm cuff is inflated exceeding the maximum allowable pressure.	Do not touch the arm cuff and/or bend the air tube while taking a measurement. If inflating the arm cuff manually, refer to section 6 of instruction manual (2).
 E4 appears	You move or talk during a measurement. Vibrations disrupt a measurement.	Remain still and do not talk during a measurement.
 E5 appears	The pulse rate is not detected correctly.	Apply the arm cuff correctly, then take another measurement. Refer to section 4 of instruction manual (2).
 appears		Remain still and sit correctly during a measurement.
 does not flash during a measurement		If the "  " symbol continues to appear, we recommend you to consult with your physician.

Display/Problem	Possible Cause	Solution
 appears	The monitor has malfunctioned.	Press the [START/STOP] button again. If “Er” still appears, contact your OMRON retail outlet or distributor.
 appears	Batteries are low.	Replacing all 4 batteries with new ones is recommended. Refer to section 3 of instruction manual (2).
 appears or the monitor is turned off unexpectedly during a measurement	Batteries are depleted.	Immediately replace all 4 batteries with new ones. Refer to section 3 of instruction manual (2).
Nothing appears on the display of the monitor.	Battery polarities are not properly aligned.	Check the battery installation for proper placement. Refer to section 3 of instruction manual (2).
Readings appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 2 of instruction manual (2).	
Any other problem occurs.	Press the [START/STOP] button to turn the monitor off, then press it again to take a measurement. If the problem continues, remove all batteries and wait for 30 seconds. Then re-install batteries. If the problem still persists, contact your OMRON retail outlet or distributor.	

4. Limited Warranty

Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing. It is designed to give you every satisfaction, provided that it is properly operated and maintained as described in the instruction manual.

This product is warranted by OMRON for a period of 5 years after the date of purchase. The proper construction, workmanship and materials of this product is warranted by OMRON. During this period of warranty OMRON will, without charge for labour or parts, repair or replace the defect product or any defective parts.

The warranty does not cover any of the following:

- A. Transport costs and risks of transport.
- B. Costs for repairs and / or defects resulting from repairs done by unauthorised persons.
- C. Periodic check-ups and maintenance.
- D. Failure or wear of optional parts or other attachments other than the main device itself, unless explicitly warranted above.
- E. Costs arising due to non-acceptance of a claim (those will be charged for).
- F. Damages of any kind including personal caused accidentally or from misuse.
- G. Calibration service is not included within the warranty.
- H. Optional parts have a one (1) year warranty from date of purchase. Optional parts include, but are not limited to the following items: cuff and cuff tube.

Should warranty service be required please apply to the dealer whom the product was purchased from or an authorised OMRON distributor. For the address refer to the product packaging / literature or to your specialised retailer. If you have difficulties in finding OMRON customer services, contact us for information:
www.omron-healthcare.com

Repair or replacement under the warranty does not give rise to any extension or renewal of the warranty period.

The warranty will be granted only if the complete product is returned together with the original invoice / cash ticket issued to the consumer by the retailer.

5. Maintenance

5.1 Maintenance

To protect your monitor from damage, follow the directions below: Changes or modifications not approved by the manufacturer will void the user warranty.

Caution

DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

5.2 Storage

- Store your monitor and other components in a clean, safe location.
- Do not store your monitor and other components:
 - If your monitor and other components are wet.
 - In locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
 - In locations exposed to vibrations or shocks.

5.3 Cleaning

- Do not use any abrasive or volatile cleaners.
- Use a soft dry cloth or a soft cloth moistened with mild (neutral) detergent to clean your monitor and arm cuff, and then wipe them with a dry cloth.
- Do not wash or immerse your monitor and arm cuff or other components in water.
- Do not use gasoline, thinners or similar solvents to clean your monitor and arm cuff or other components.

5.4 Calibration and Service

- The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
- It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON Customer Service at the address given on the packaging or attached literature.

6. Specifications

Product description	Automatic Upper Arm Blood Pressure Monitor		
Product category	Electronic Sphygmomanometers		
Model (code)	M2 Basic (HEM-7121J-E) X2 Basic (HEM-7121J-EO)	Display	LCD digital display
Cuff pressure range	0 to 299 mmHg	Pulse measurement range	40 to 180 beats / min.
Blood pressure measurement range	20 to 280 mmHg		
Accuracy	Pressure: ± 3 mmHg / Pulse: $\pm 5\%$ of display reading		
Inflation	Automatic by electric pump	Deflation	Automatic pressure release valve
Measurement method	Oscillometric method	Operating mode	Continuous operation
IP classification	Monitor: IP20 / Optional AC adapter: IP21 (HHP-CM01) or IP22 (HHP-BFH01)		
Rating	DC6 V 4.0 W	Applied part	Type BF (arm cuff)
Power source	4 "AA" batteries 1.5 V or optional AC adapter (INPUT AC 100 - 240 V 50 - 60 Hz 0.12 - 0.065 A)		
Battery life	Approximately 1000 measurements (using new alkaline batteries)		
Durable period (Service life)	Monitor: 5 years / Cuff: 5 years / Optional AC adapter: 5 years		
Operating conditions	+10 to +40 °C / 15 to 90% RH (non-condensing) / 800 to 1060 hPa		
Storage / Transport conditions	-20 to +60 °C / 10 to 90% RH (non-condensing)		
Contents	Monitor, arm cuff (HEM-CR24), 4 "AA" batteries, Instruction Manual (1) and (2)		
Protection against electric shock	Internally powered ME equipment (when using only batteries) Class II ME equipment (optional AC adapter)		
Weight	Monitor: approximately 250 g (not including batteries) / Arm cuff: approximately 130 g		
Dimensions (approximately value)	Monitor: 108 mm (W) × 83 mm (H) × 140 mm (L) Arm cuff: 145 mm × 466 mm (air tube: 610 mm)		
Cuff circumference applicable to the monitor	170 to 420 mm (included arm cuff: (220 to 320 mm))	Maximum temperature of the applied part	Lower than 48 °C
Memory	Last measurement		

Note

- These specifications are subject to change without notice.
- This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2014 and EN ISO 81060-2:2019 + A1:2020 (excluding pregnant and pre-eclampsia patients). In the clinical validation study, K5 was used on 85 subjects for determination of diastolic blood pressure.
- IP classification is degrees of protection provided by enclosures in accordance with IEC 60529. This monitor and optional AC adapter are protected against solid foreign objects of 12.5 mm diameter and greater such as a finger. The optional AC adapter HHP-CM01 is protected against vertically falling water drops which may cause issues during a normal operation. The optional AC adapter HHP-BFH01 is protected against oblique falling water drops which may cause issues during a normal operation.
- Operating mode is classification in accordance with IEC 60601-1.

7. Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life.

To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.



8. Important Information Regarding Electromagnetic Compatibility (EMC)

HEM-7121J-E conforms to EN60601-1-2:2015 Electromagnetic Compatibility (EMC) standard.

Further documentation in accordance with this EMC standard is available at www.omron-healthcare.com

Refer to the EMC information for HEM-7121J-E on the website.

9. Guidance and Manufacturer's Declaration

- This blood pressure monitor is designed according to the European Standard EN1060, Non-invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.
- This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., Japan. The Core component for OMRON blood pressure monitors, which is the Pressure Sensor, is produced in Japan.
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

10. Classification of BP (Blood Pressure)

2018 ESH/ESC* Guidelines for the management of arterial hypertension

Definitions of hypertension by office and home blood pressure levels

	Office	Home
Systolic Blood Pressure	≥ 140 mmHg	≥ 135 mmHg
Diastolic Blood Pressure	≥ 90 mmHg	≥ 85 mmHg

These ranges are from statistical values for blood pressure.

* European Society of Hypertension (ESH) and European Society of Cardiology (ESC).

⚠ Warning

NEVER diagnose or treat yourself based on your readings.

ALWAYS consult with your physician.

Symbols Description

FR Description des symboles
DE Beschreibung der Symbole
IT Descrizione dei simboli
ES Descripción de los símbolos

RU Описание символов
TR Simgelerin Açıklaması
PL Opis symboli
PT Descrição dos símbolos

شرح الرموز **AR**



Applied part - Type BF Degree of protection against electric shock (leakage current)

FR Pièce appliquée - Type BF Degré de protection contre les chocs électriques (courant de fuite)

DE Anwendungsteil – Typ BF Schutz vor Stromschlägen (Ableitstrom)

IT Parti applicate - Tipo BF Livello di protezione contro le folgorazioni (corrente di dispersione)

ES Partes en contacto: Tipo BF Grado de protección contra descargas eléctricas (corriente de fuga)

RU Рабочая часть аппарата - Степень защиты от поражения электрическим током (токи утечки) Тип BF

TR Uygulanan parça - Tip BF Elektrik çarpmasına karşı koruma derecesi (kaçak akım)

PL Część wchodząca w kontakt z ciałem pacjenta — typu BF (stopień ochrony przed porażeniem prądem (prąd upływu))

PT Parte aplicada - Tipo BF Grau de proteção contra choques elétricos (corrente de fuga)

الجزء المطبق: درجة الحماية
 ضد الصدمات الكهربائية (التيار
 المتسرب) للنوع BF **AR**



Class II equipment.

Protection against electric shock

FR Équipement de classe II. Protection contre les chocs électriques

DE Gerät der Klasse II. Schutz vor Stromschlägen

IT Apparecchiatura di Classe II. Protezione contro le folgorazioni

ES Equipo de Clase II. Protección contra descargas eléctricas

RU Изделие класса II. Защита от поражения электрическим током

TR Sınıf II ünite. Elektrik çarpmasına karşı koruma

PL "Urządzenie klasy II. Ochrona przed porażeniem prądem"

PT Equipamento da Classe II. Proteção contra choques elétricos

جهاز من الفئة II الحماية ضد
 الصدمات الكهربائية **AR**

IP XX

Ingress protection degree provided by IEC 60529

FR Degré de protection selon CEI 60529

DE Grad des Eindring-schutzes gemäß IEC 60529

IT Livello di protezione IP in base a IEC 60529

ES Grado de protección según la norma internacional IEC 60529





RU Степень защиты, обеспечиваемая оболочками, согласно IEC 60529





TR Su girmesine karşı koruma derecesi IEC 60529 tarafından verilmiştir




PL Stopień ochrony wg IEC 60529



PT Grau de proteção contra entradas indicado pela IEC 60529




درجة حماية الدخول المقدمة
 بواسطة معيار IEC 60529 **AR**


	<p>CE Marking</p> <p>FR Marquage CE DE CE-Kennzeichnung IT Contrassegno CE ES Marcado CE</p> <p>RU Знак соответствия директиве ЕС TR CE İşareti PL Oznaczenie CE PT Marca CE</p> <p>AR علامة التوافق مع اللجنة الأوروبية (CE)</p>
	<p>UKCA Marking</p> <p>FR Marquage UKCA DE UKCA-Kennzeichnung IT Marchio UKCA ES Marcado UKCA</p> <p>RU Маркировка UKCA TR UKCA İşareti PL Oznaczenie UKCA PT Marca UKCA</p> <p>AR UKCA علامة</p>
	<p>Serial number</p> <p>FR Numéro de série DE Seriennummer IT Numero di serie ES Número de serie</p> <p>RU Серийный номер TR Seri numarası PL Numer serii PT Número de série</p> <p>AR الرقم المتسلسل</p>
	<p>LOT number</p> <p>FR Numéro de LOT DE LOT-Nummer IT Numero di lotto ES Número de lote</p> <p>RU Код (номер) партии TR Parti numarası PL Numer partii PT Número de LOTE</p> <p>AR رقم التشغيلية</p>

	<p>Unique device identifier</p> <p>FR Identification unique des dispositifs DE Produktidentifizierungsnummer IT Identificatore univoco del dispositivo ES Identificador único del dispositivo</p> <p>RU Уникальный идентификатор устройства TR Benzersiz cihaz tanımlayıcısı PL Unikatowy identyfikator urządzenia PT Identificador único do dispositivo</p> <p>AR معرف الجهاز الفريد</p>
	<p>Medical device</p> <p>FR Dispositif médical DE Medizinprodukt IT Dispositivo medico ES Producto sanitario</p> <p>RU Медицинский прибор TR Tibbi cihaz PL Wyrób medyczny PT Dispositivo médico</p> <p>AR جهاز طبي</p>
	<p>Temperature limitation</p> <p>FR Limitation de température DE Temperaturbegrenzung IT Limite di temperatura ES Limitación de la temperatura</p> <p>RU Температурный диапазон TR Sıcaklık sınırlaması PL Ograniczenia dot. temperatury PT Limite de temperatura</p> <p>AR حدود درجة الحرارة المناسبة</p>
	<p>Humidity limitation</p> <p>FR Limitation d'humidité DE Luftfeuchtigkeitsbegrenzung IT Limite di umidità ES Limitación de la humedad</p> <p>RU Диапазон влажности TR Nem sınırlaması PL Ograniczenia dot. wilgotności PT Limite de humidade</p> <p>AR حدود الرطوبة المناسبة</p>

	<p>Atmospheric pressure limitation</p> <p>FR Limitation de pression atmosphérique</p> <p>DE Luftdruckbegrenzung</p> <p>IT Limite di pressione atmosferica</p> <p>ES Limitación de la presión atmosférica</p> <p>RU Ограничение атмосферного давления</p> <p>TR Atmosferik basınç sınırlaması</p> <p>PL Ograniczenia ciśnienia atmosferycznego</p> <p>PT Limite de pressão atmosférica</p> <p>AR حدود الضغط الجوي المناسب</p>
	<p>Indication of connector polarity</p> <p>FR Indication de la polarité des connecteurs</p> <p>DE Anzeige der Steckerpolarität</p> <p>IT Indicazione della polarità dei connettori</p> <p>ES Indicación de la polaridad del conector</p> <p>RU Полярность разъема адаптера</p> <p>TR Bağlantı polarite göstergesi</p> <p>PL Oznaczenie biegunowości złącza</p> <p>PT Indicação da polaridade do conector</p> <p>AR علامة تشير لقطبية الموصل</p>
	<p>For indoor use only</p> <p>FR Pour un usage à l'intérieur uniquement</p> <p>DE Nur für die Nutzung in Innenbereichen</p> <p>IT Solo per uso in interni</p> <p>ES Para uso solo en interiores</p> <p>RU Для использования только внутри помещений</p> <p>TR Sadece iç mekanda kullanım için</p> <p>PL Wyłącznie do użytku wewnętrznego</p> <p>PT Apenas para utilização em interior</p> <p>AR صالح للاستخدام في المناطق المغلقة فقط</p>

 <p>Intelli sense</p>	<p>OMRON's trademarked technology for blood pressure measurement</p> <p>FR Technologie protégée par la marque de commerce OMRON pour la mesure de la pression artérielle</p> <p>DE Markenrechtlich geschützte Technologie von OMRON zur Blutdruckmessung</p> <p>IT Tecnologia brevettata OMRON per la misurazione della pressione arteriosa</p> <p>ES La tecnología de OMRON para medir la presión arterial</p> <p>RU Зарегистрированная технология измерения артериального давления OMRON</p> <p>TR OMRON'un kan basıncı ölçümü için ticari markalı teknolojisi</p> <p>PL Opatrzona znakiem towarowym technologia firmy Omron dotycząca pomiaru ciśnienia krwi</p> <p>PT Tecnologia protegida por marca comercial da OMRON para a medição da tensão arterial</p> <p>AR تقنية العلامة التجارية OMRON لقياس ضغط الدم</p>
	<p>Identifier of cuffs compatible for the device</p> <p>FR Identificateur des brassards compatibles avec l'appareil</p> <p>DE Kennzeichnung der mit dem Gerät kompatiblen Manschetten</p> <p>IT Identifica i bracciali compatibili con il dispositivo</p> <p>ES Identificador para manguitos compatibles con el dispositivo</p> <p>RU Манжеты совместимые с устройством</p> <p>TR Cihaz ile uyumlu kollukların tanıtım işareti</p> <p>PL Sposób identyfikacji mankietów zgodnych z urządzeniem</p> <p>PT Identificador de braçadeiras compatíveis com o dispositivo</p> <p>AR علامات تحديد الشرايط الضاغطة المتوافقة مع الجهاز</p>

	<p>Cuff positioning indicator for the left arm</p> <p>FR Indicateur de positionnement du brassard pour le bras gauche</p> <p>DE Indikator für die Manschettenposition am linken Arm</p> <p>IT Indicatore di posizionamento del bracciale per il braccio sinistro</p> <p>ES Indicador de posición del manguito en el brazo izquierdo</p> <p>RU Метка для правильного расположения манжеты на левой руке</p> <p>TR Sol kol için kolluk konumlandırma göstergesi</p> <p>PL Wskaźnik umiejscowienia mankieta na lewym ramieniu</p> <p>PT Indicador de posicionamento da braçadeira no braço esquerdo</p> <p>AR المؤشر الخاص بموضع الشريط الضاغط للذراع الأيسر</p>
<p>ART.</p>  	<p>Marker on the cuff to be positioned above the artery</p> <p>FR Repère sur le brassard, à positionner au-dessus de l'artère</p> <p>DE Markierung auf der Manschette, die oberhalb der Arterie liegen muss</p> <p>IT Contrassegno sul bracciale da posizionare al di sopra dell'arteria</p> <p>ES La marca del manguito debe colocarse sobre la arteria</p> <p>RU Указатель расположения плечевой артерии</p> <p>TR Kolluk üzerindeki işaretin konumu arterin üzerine gelmelidir</p> <p>PL Znacznik na mankiecie, wskazujący pozycję umieszczenia nad tętnicą</p> <p>PT Marcador da braçadeira a posicionar sobre a artéria</p> <p>AR علامة بالشريط الضاغط للإشارة إلى وجوب وضعه فوق الشريان</p>

<p>INDEX</p> 	<p>Range pointer and brachial artery alignment position</p> <p>FR Pointeur de plage et position d'alignement sur l'artère brachiale</p> <p>DE Bereichsanzeiger und Ausrichtungsposition mit der Oberarmarterie</p> <p>IT Puntatore e posizione di allineamento dell'arteria brachiale</p> <p>ES Indicación de las medidas y posición de alineación con la arteria braquial</p> <p>RU Положение совмещения указателя диапазона и плечевой артерии</p> <p>TR Aralık işaretçisi ve kol arteri hizalama konumu</p> <p>PL Wskaźnik zakresu i dostosowania pozycji do tętnicy ramiennej</p> <p>PT Indicador de gama e posição de alinhamento da artéria braquial</p> <p>AR مؤشر النطاق والوضع الخاص بالمحاذاة مع الشريان العضدي</p>



Range indicator of arm circumferences to help selection of the correct cuff size

FR Indicateur de plage de circonférence du bras, pour la sélection de la taille de brassard adaptée

DE Bereichsanzeige für den Armumfang zur Auswahl der richtigen Manschettengröße

IT Indicatore degli intervalli di circonferenze braccio per la scelta della misura di bracciale corretta

ES Indicador de las medidas del perímetro de brazo para ayudarle a seleccionar el tamaño de manguito correcto

RU Символ диапазона окружности плеча для помощи при подборе правильного размера манжеты

TR Doğru kolluk ölçüsünün bulmaya yardımcı olan kol çevresi aralık göstergesi

PL Wskaźnik zakresu obwodu ramienia pomagający w doborze mankietu o właściwym rozmiarze

PT Indicador da gama de perímetros do braço que auxilia a seleção do tamanho de braçadeira correto

AR مؤشر نطاق محيط الذراع للمساعدة على تحديد الحجم الصحيح للشريط الضاغط

QUALITY PASS



Manufacturer's quality control mark

FR Marque de contrôle de la qualité du fabricant

DE Qualitätskontrollzeichen des Herstellers

IT Contrassegno controllo qualità del produttore

ES Marca del control de calidad del fabricante

RU Отметка производителя о контроле качества

TR Üreticinin kalite kontrol işareti

PL Znak kontroli jakości producenta

PT Marca de controlo da qualidade do fabricante

AR علامة التحكم في الجودة الخاصة بالشركة المصنعة

LATEX FREE

Not made with natural rubber latex

FR Ne contient pas de latex de caoutchouc naturel

DE Enthält kein Naturlatex

IT Non contiene lattice di gomma naturale

ES No contiene látex de caucho natural

RU Не содержит натуральный латекс

TR Doğal kauçuk lateksten üretilmemiştir

PL Wyprodukowane bez użycia naturalnego lateksu

PT Não é fabricado em látex de borracha natural

AR تم التصنيع دون استخدام لاتيكس المطاط الطبيعي



Arm circumference

FR Circonférence du bras

DE Armumfang

IT Circonferenza del braccio

ES Perímetro de brazo

RU Окружность плеча

TR Kol çevresi

PL Obwód ramienia

PT Circunferência do braço

AR محيط الذراع



Necessity for the user to consult this instruction manual

FR L'utilisateur doit consulter le présent mode d'emploi

DE Der Benutzer muss diese Gebrauchs-

anweisung lesen

IT L'utente deve consultare il presente manuale di istruzioni

ES Es necesario que el usuario consulte este manual de instrucciones

RU Необходимость для пользователя обратиться к данному руководству по эксплуатации

TR Kullanıcı, bu kullanım kılavuzuna başvurmalıdır

PL Użytkownik powinien zapoznać się z niniejszą instrukcją obsługi.

PT O utilizador tem de consultar este manual de instruções

AR ضرورة رجوع المستخدم إلى المعلومات الواردة في دليل الإرشادات هذا



Need for the user to follow this instruction manual thoroughly for your safety.

FR L'utilisateur doit suivre attentivement ce mode d'emploi pour votre sécurité.

DE Damit die Sicherheit gewährleistet ist, muss der Benutzer diese Gebrauchsanweisung sorgfältig befolgen.

IT Per la propria sicurezza, l'utente deve seguire attentamente il presente manuale di istruzioni.

ES Es necesario que el usuario siga rigurosamente este manual de instrucciones para su seguridad.

RU В целях обеспечения безопасности строго следуйте указаниям в данном руководстве по эксплуатации.

TR Güvenlik açısından kullanıcının bu kullanım kılavuzuna dikkatle uyması gerekir.

PL Dla zachowania bezpieczeństwa użytkownik musi ściśle przestrzegać niniejszej instrukcji obsługi.

PT O utilizador tem de seguir cuidadosamente este manual de instruções para sua própria segurança.

AR حاجة المستخدم إلى اتباع الإرشادات الواردة في دليل الإرشادات بالكامل هذا للحفاظ على السلامة.



Direct current

FR Courant continu

DE Gleichstrom

IT Corrente diretta

ES Corriente directa




RU Постоянный ток

TR Doğru akım

PL Prąd stały

PT Corrente direta

AR التيار المباشر

	<p>Alternating current</p> <table border="0"> <tr> <td>FR Courant alternatif</td> <td>RU Переменный ток</td> </tr> <tr> <td>DE Wechselstrom</td> <td>TR Alternatif akım</td> </tr> <tr> <td>IT Corrente alternata</td> <td>PL Prąd zmienny</td> </tr> <tr> <td>ES Corriente alterna</td> <td>PT Corrente alterna</td> </tr> </table> <p style="text-align: right;">التيار المتناوب AR</p>	FR Courant alternatif	RU Переменный ток	DE Wechselstrom	TR Alternatif akım	IT Corrente alternata	PL Prąd zmienny	ES Corriente alterna	PT Corrente alterna
FR Courant alternatif	RU Переменный ток								
DE Wechselstrom	TR Alternatif akım								
IT Corrente alternata	PL Prąd zmienny								
ES Corriente alterna	PT Corrente alterna								
	<p>Date of manufacture</p> <table border="0"> <tr> <td>FR Date de fabrication</td> <td>RU Дата изготовления</td> </tr> <tr> <td>DE Herstellungsdatum</td> <td>TR Üretim tarihi</td> </tr> <tr> <td>IT Data di fabbricazione</td> <td>PL Data produkcji</td> </tr> <tr> <td>ES Fecha de fabricación</td> <td>PT Data de fabrico</td> </tr> </table> <p style="text-align: right;">تاريخ التصنيع AR</p>	FR Date de fabrication	RU Дата изготовления	DE Herstellungsdatum	TR Üretim tarihi	IT Data di fabbricazione	PL Data produkcji	ES Fecha de fabricación	PT Data de fabrico
FR Date de fabrication	RU Дата изготовления								
DE Herstellungsdatum	TR Üretim tarihi								
IT Data di fabbricazione	PL Data produkcji								
ES Fecha de fabricación	PT Data de fabrico								
	<p>Prohibited action</p> <table border="0"> <tr> <td>FR Action interdite</td> <td>RU Запрещенные действия</td> </tr> <tr> <td>DE Verbotene Aktion</td> <td>TR Yasaklanmış eylem</td> </tr> <tr> <td>IT Operazione proibita</td> <td>PL Czynność niedozwolona</td> </tr> <tr> <td>ES Acción prohibida</td> <td>PT Ação proibida</td> </tr> </table> <p style="text-align: right;">إجراء محظور AR</p>	FR Action interdite	RU Запрещенные действия	DE Verbotene Aktion	TR Yasaklanmış eylem	IT Operazione proibita	PL Czynność niedozwolona	ES Acción prohibida	PT Ação proibida
FR Action interdite	RU Запрещенные действия								
DE Verbotene Aktion	TR Yasaklanmış eylem								
IT Operazione proibita	PL Czynność niedozwolona								
ES Acción prohibida	PT Ação proibida								

Issue Date:
Date de publication :
Ausgabedatum:
Data di pubblicazione:
Fecha de publicación: 2023-10-24
Дата выпуска:
Teslim Tarihi:
Data publikacji:
Data de edição:
تاريخ الإصدار:

IM1-HEM-7121J-E-05-04/2023
3275340-4E

Automatic Upper Arm
Blood Pressure Monitor
M2 Basic (HEM-7121J-E)
X2 Basic (HEM-7121J-EO)

All for Healthcare



IM2-HEM-7121J-E-03-04/2022
3275343-9C

Read instruction manual **1** and **2** before use.


- FR** Lire le mode d'emploi **1** et **2** avant l'utilisation.
- DE** Lesen Sie vor der Verwendung die Gebrauchsanweisung **1** und **2**.
- IT** Leggere il manuale di istruzioni **1** e **2** prima dell'uso.
- ES** Lea el manual de instrucciones **1** y **2** antes del uso.
- RU** Прочтите руководство по эксплуатации **1** и **2** перед использованием.
- TR** Kullanmadan önce, kullanım kılavuzu **1** ve **2**'yi okuyun.
- PL** Przed rozpoczęciem korzystania z ciśnieniomierza należy przeczytać instrukcję obsługi **1** i **2**.
- PT** Leia o Manual de instruções **1** e **2** antes de utilizar.

AR اقرأ دليل الإرشادات **1** و **2** قبل الاستخدام.

1 Package Contents

- FR** Contenu de l'emballage
- DE** Packungsinhalt
- IT** Contenuto della confezione
- ES** Contenido del envase
- RU** Комплект поставки
- TR** Paketin İçindekiler
- PL** Zawartość opakowania
- PT** Conteúdo da embalagem

AR محتويات العبوة

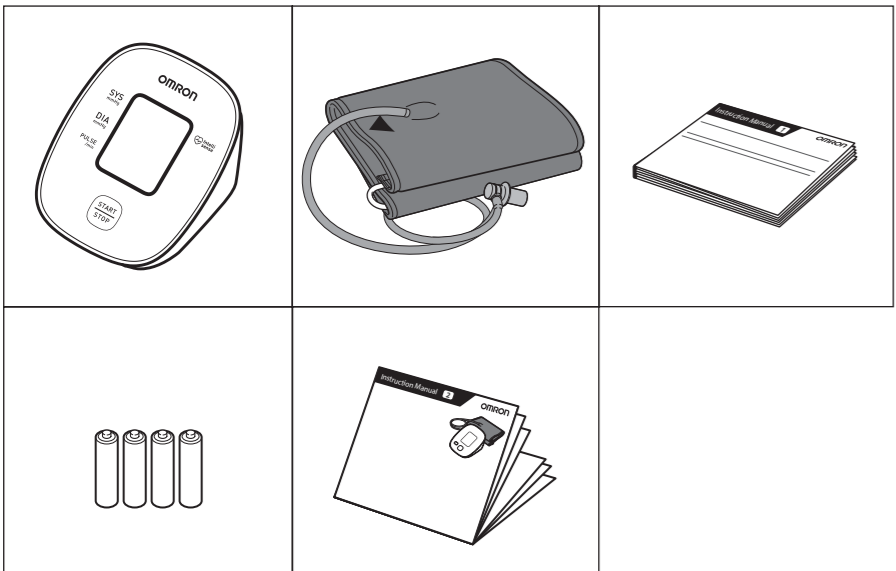

2 Preparing for a Measurement

- FR** Préparation d'une mesure
- DE** Vorbereiten einer Messung
- IT** Preparazione per la misurazione
- ES** Preparación para una medición
- RU** Подготовка к измерению
- TR** Ölçüm Hazırlığı
- PL** Przygotowanie do pomiaru
- PT** Preparação de uma medição

AR التجهيز لعملية القياس

30 minutes before

- FR** 30 minutes avant
- DE** 30 Minuten vorher
- IT** 30 minuti prima
- ES** 30 minutos antes
- RU** За 30 минут до
- TR** 30 dakika önce
- PL** 30 minut przed
- PT** 30 minutos antes

AR قبل القياس بمدة تبلغ ٣٠ دقيقة



5 minutes before: Relax and rest.

- FR** 5 minutes avant : détente et repos.
- DE** 5 Minuten vorher: ruhig hinsetzen.
- IT** 5 minuti prima: rilassarsi e stare a riposo.
- ES** 5 minutos antes: relájese y descanse.
- RU** За 5 минут до: расслабьтесь и отдохните.
- TR** 5 dakika önce: Gevşeyin ve dinlenin.
- PL** 5 minut przed: odpręż się i odpocznij.
- PT** 5 minutos antes: descontrair e repousar.


AR قبل القياس بمدة تبلغ ٥ دقائق: استرخ واسترح.



3 Installing Batteries

- FR** Mise en place des piles
- DE** Einsetzen der Batterien
- IT** Installazione delle batterie
- ES** Instalación de las pilas
- RU** Установка элементов питания
- TR** Pillerin Takılması
- PL** Instalacja baterii
- PT** Instalação das pilhas


AR تركيب البطاريات

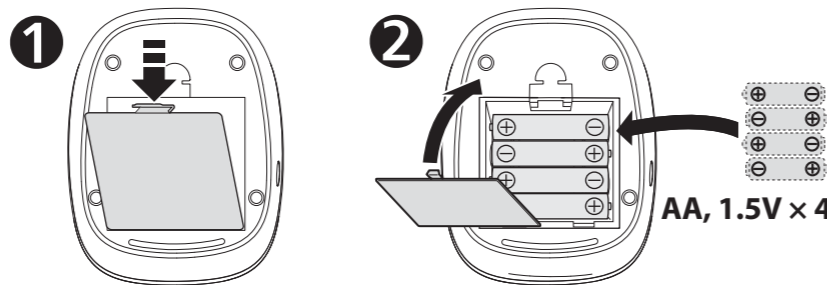

4 Applying the Cuff on the Left Arm

- FR** Pose du brassard sur le bras gauche
- DE** Anbringen der Manschette am linken Arm
- IT** Applicazione del bracciale sul braccio sinistro
- ES** Colocación del manguito en el brazo izquierdo
- RU** Расположение манжеты на левой руке
- TR** Koluğun Sol Kola Takılması
- PL** Zakładanie mankietu na lewe ramię
- PT** Aplicação da braçadeira no braço esquerdo

AR لف الشريط الضاغط على الذراع الأيمن

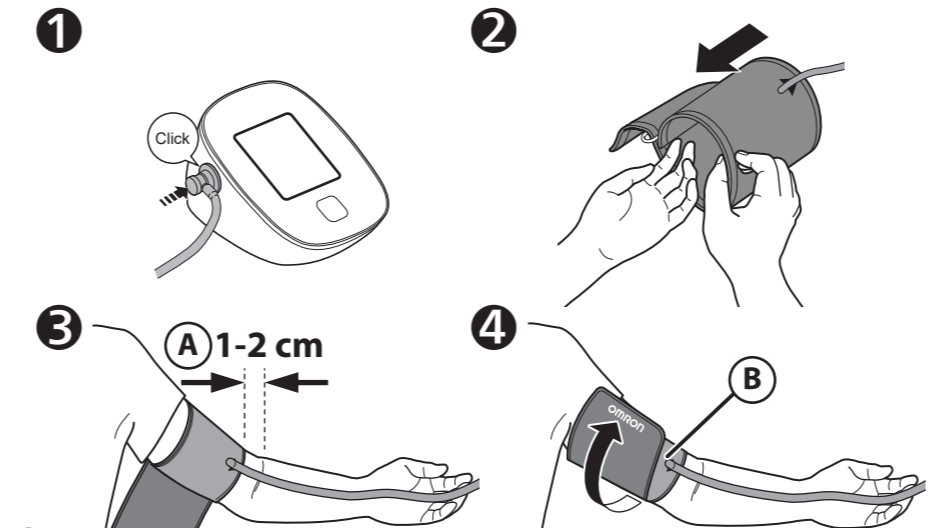

A Tube side of the cuff should be 1 - 2 cm above the inside elbow.

- FR** Le côté tuyau du brassard doit être positionné 1 à 2 cm au-dessus de l'intérieur du coude.
- DE** Das Manschettenstück mit dem Schlauch muss 1 bis 2 cm oberhalb des Ellbogens liegen.
- IT** Il lato del bracciale con il tubo deve trovarsi al di sopra dell'interno del gomito, a una distanza di circa 1 o 2 cm.
- ES** El lado del tubo del manguito deberá quedar 1 o 2 cm por encima de la parte interna del codo.
- RU** Край манжеты с трубкой должен находиться на 1–2 см выше локтевого сгиба.
- TR** Koluğun boru tarafı, dirsek içinin 1 ila 2 cm üstünde olmalıdır.
- PL** Koniec mankietu z podłączonym przewodem powietrza powinien znajdować się 1–2 cm powyżej zgięcia łokcia.
- PT** O lado do tubo da braçadeira deve estar 1 - 2 cm acima do interior do cotovelo.

AR يجب أن يرتفع جانب الخرطوم من الشريط الضاغط من ١ إلى ٢ سم عن المرفق.

B Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

- FR** Assurez-vous que le tuyau à air se trouve du côté intérieur de votre bras et enroulez fermement le brassard de manière qu'il ne puisse plus tourner.
- DE** Stellen Sie sicher, dass der Luftschlauch an der Arminnenseite sitzt, und befestigen Sie die Manschette sicher, so dass sie nicht verrutscht.
- IT** Assicurarsi che il tubo dell'aria si trovi all'interno del braccio e avvolgere il bracciale saldamente in modo che non possa ruotare.
- ES** Asegúrese de que el tubo de aire se encuentra en la cara interna del brazo y enrole el manguito con firmeza para que no pueda deslizarse.
- RU** Убедитесь, что воздуховодная трубка находится на внутренней поверхности плеча и надежно оберните манжету, чтобы она не прокручивалась вокруг руки.
- TR** Hava borusunun kolunuzun iç tarafında olduğundan emin olun ve koluğu kaunmayacak şekilde sabit şekilde sarın.
- PL** Upewnij się, że przewód powietrza znajduje się po wewnętrznej stronie ramienia i owinąć starannie mankiet, aby się nie zsuwał.
- PT** Certifique-se de que o tubo de ar está na parte interior do braço e enrole a braçadeira firmemente, para que não deslize.

If taking measurements on the right arm, refer to:

- FR** Pour la prise de mesures au bras droit, voir :
- DE** Bei Messungen am rechten Arm siehe:
- IT** Se la misurazione viene eseguita al braccio destro, fare riferimento a:
- ES** Si va a realizar mediciones en el brazo derecho, consulte:
- RU** При измерении на правой руке см:
- TR** Sağ koldan ölçüm yapıyorsanız aşağıdakilere bakın:
- PL** Pomiar ciśnienia na prawym ramieniu, patrz:
- PT** Se fizer as medições no braço direito, consulte:




Instruction Manual **1**
2.3

AR في حال إجراء القياس على الذراع الأيمن، ارجع إلى:

5 Sitting Correctly

- FR** Position assise correcte
- DE** Korrekte Körperhaltung
- IT** Come sedere nel modo corretto
- ES** Cómo sentarse correctamente
- RU** Сядьте правильно
- TR** Düzgün Oturma
- PL** Zakładanie mankietu na lewe ramię
- PT** Como sentar-se corretamente

AR الجلوس بشكل صحيح

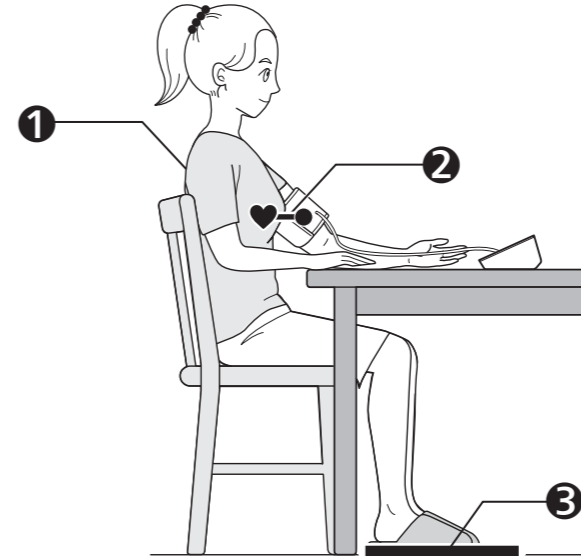

1 Sit comfortably with your back and arm supported.

- FR** S'asseoir de façon à ce que le dos et le bras soient bien soutenus.
- DE** Mit dem Rücken anlehnen und den Arm auf eine Unterlage legen.
- IT** Sedere con la schiena e il braccio ben sostenuti.
- ES** Siéntese de modo que la espalda y el brazo estén bien apoyados.
- RU** Сядьте так, чтобы спина и рука опирались на что-либо.
- TR** Sirtınız ve kolunuz desteklenecek şekilde oturun.
- PL** Usiąść wygodnie, opierając plecy i ramię.
- PT** Sente-se confortavelmente apoiando as costas e o braço.

AR اجلس بحيث يكون الظهر والذراع في وضع استناد.

2 Place the arm cuff at the same level as your heart.

- FR** Le brassard doit se trouver au même niveau que votre cœur.
- DE** Die Manschette auf Herzhöhe platzieren.
- IT** Posizionare il bracciale allo stesso livello del cuore.
- ES** Coloque el manguito al mismo nivel que el corazón.
- RU** Манжета должна находиться на уровне сердца.
- TR** Koluğu kalbinizle aynı düzeye getirin.
- PL** Mankiet powinien znajdować się na wysokości serca.
- PT** Coloque a braçadeira ao mesmo nível do coração.

AR ضع الشريط الضاغط للذراع في مستوى القلب نفسه.

3 Keep feet flat, legs uncrossed, remain still and do not talk.


- FR** Garder les pieds à plat, les jambes non croisées, ne pas bouger et ne pas parler.
- DE** Füße flach auf den Boden stellen, die Beine nicht kreuzen, still halten und nicht sprechen.
- IT** Tenere i piedi ben poggiati, non incrociare le gambe, rimanere fermi e non parlare.
- ES** Mantenga los pies planos, las piernas sin cruzar, quédese quieto y no hable.
- RU** Поставьте ноги ровно, не перекрещивая их, не двигайтесь и не разговаривайте.
- TR** Ayaklarınızı düz, bacaklarınızı açık tutun, hareketsiz kalın ve konuşmayın.
- PL** Stopy ułożyć płasko na podłodze, nie krzyżować nóg, siedzieć spokojnie, bez ruchu i nie rozmawiać.
- PT** Mantenha os pés no chão, não cruze as pernas, mantenha-se imóvel e não fale.

AR اجعل قدميك في وضع مسطح وساقيك في وضع متباعد وحافظ على ثباتك ولا تتحدث.

6 Taking a Measurement

- FR** Réalisation d'une mesure
- DE** Vornehmen einer Messung
- IT** Misurazione
- ES** Obtención de una lectura
- RU** Выполнение измерений
- TR** Ölçüm Yapma
- PL** Prawidłowa pozycja ciała
- PT** Realização de uma medição

AR إجراء القياس

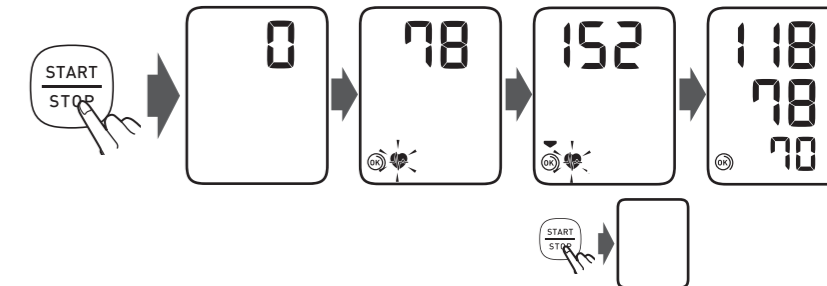

When the [START/STOP] button is pressed, the measurement is taken and saved automatically.

- FR** Après une pression sur le bouton [START/STOP], la mesure démarre et le résultat est enregistré automatiquement.
- DE** Beim Drücken der Taste [START/STOP] erfolgt die Messung. Die Messwerte werden automatisch gespeichert.
- IT** Quando si preme il pulsante [START/STOP], la misurazione viene eseguita e salvata automaticamente.
- ES** Al pulsar el botón [START/STOP], se obtiene la medición, que se guarda automáticamente.
- RU** После того, как кнопка [START/STOP] нажата, измерение выполняется и сохраняется автоматически.
- TR** [START/STOP] düğmesine basıldığında ölçüm yapılır ve otomatik olarak kaydedilir.
- PL** Po naciśnięciu przycisku [START/STOP] następuje pomiar i jego automatyczny zapis.
- PT** Quando o botão [START/STOP] é premido, a medição é feita e gravada automaticamente.

AR عند الضغط على زر [START/STOP]، يتم إجراء القياس وحفظه تلقائيًا.

If your systolic pressure is more than 210 mmHg:

After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

- FR** Si votre pression systolique est supérieure à 210 mmHg :
- DE** Wenn Ihr systolischer Druck höher ist als 210 mmHg:
- IT** Se la pressione sistolica è superiore a 210 mmHg:
- ES** En caso de que su presión arterial sistólica esté por encima de 210 mmHg:
- RU** Если систолическое давление выше 210 мм.рт.ст.:
- TR** Sistolik basıncınız 210 mmHg'den fazlaysa:
- PL** Jeżeli ciśnienie skurczowe jest wyższe niż 210 mmHg:
- PT** Se a tensão sistólica estiver acima dos 210 mmHg:

- Lorsque le brassard se gonfle, appuyer sur le bouton [START/STOP] et le maintenir enfoncé jusqu'à ce que le tensiomètre atteigne une pression de gonflage supérieure de 30 à 40 mmHg à votre pression systolique attendue.
- Nachdem die Manschette begonnen hat, sich aufzupumpen, drücken Sie die [START/STOP]-Taste, und halten Sie sie gedrückt, bis ein Druck von 30 bis 40 mmHg über Ihrem erwarteten systolischen Druck erreicht ist.
- Quando il bracciale inizia a gonfiarsi, premere e mantenere premuto il pulsante [START/STOP] finché il misuratore non raggiunge una pressione da 30 a 40 mmHg superiore rispetto al valore di pressione sistolica atteso.
- Una vez que el manguito comienza a inflarse, mantenga pulsado el botón [START/STOP] hasta que el monitor indique que el inflado está entre 30 y 40 mmHg por encima de la presión arterial sistólica estimada.
- После того, как началось автоматическое нагнетание воздуха в манжету, нажмите и удерживайте кнопку [START/STOP] до тех пор, пока электронный блок не поднимет давление до значения, превышающего ожидаемое систолическое давление на 30–40 мм рт. ст.
- Koluğu kalbinizle aynı düzeye getirin.
- Mankiet powinien znajdować się na wysokości serca.
- Coloque a braçadeira ao mesmo nível do coração.

AR إذا كان ضغط دمك الانقباضي أكثر من ٢١٠ ملليمترات زئبقية:

بعد بدء انتفاخ الشريط الضاغط للذراع، اضغط مع الاستمرار على الزر [START/STOP] (بدء/إيقاف) حتى يقوم الجهاز بالنفخ من ٣٠ إلى ٤٠ ملليمترًا زئبقيًا أعلى من قيمة ضغط الدم الانقباضي المتوقع لديك.

