Automatic Upper Arm Blood Pressure Monitor M2 (HEM-7143-E)

Read Instruction manual 1 and 2 before use.

- Display/Problem: Monitor malfunctioned, batteries low, nothing appears on display, readings appear too high/low. Possible Cause: Monitor not plugged in, AC adapter not inserted, arm cuff not inflated, etc. Solution: Press the START/STOP button, replace batteries, check battery placement, etc.

Introduction: Thank you for purchasing the M2 Automatic Upper Arm Blood Pressure Monitor. This device is designed to give you an accurate reading of your blood pressure.

Safety Instructions: This instruction manual provides you with important information about the M2 Automatic Upper Arm Blood Pressure Monitor. Please read and understand all of these instructions.

Intended Use: This device is a digital monitor intended for use in measuring blood pressure and pulse rate to assist patient diagnosis.

Receiving and Inspection: Remove this monitor from the packaging and inspect for damage. If this monitor is damaged, DO NOT use and consult with your OMRON retail outlet or distributor.

Important Safety Information: Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety.

Warning: Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

DO NOT use this monitor on infants, toddlers, children or older adults. DO NOT adjust medication based on readings from this blood pressure monitor.

DO NOT use this monitor in areas containing high frequency radio waves. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

General Precautions

When you take a measurement on the right arm, the rest your arm should be at the side of your elbow. Be careful not to rest your arm on the air tube.

Blood pressure may differ between the right and left arm, and may result in a different measurement value. Always use the same arm for measurements.

DO NOT use this monitor in areas containing high frequency radio waves. DO NOT use this monitor in oxygen rich environments or near flammable gases.

Battery Handling and Usage

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

Battery Handling and Usage

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

Caution

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

DO NOT use this monitor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this monitor in oxygen rich environments or near flammable gases.

Table with 3 columns: Display/Problem, Possible Cause, Solution. Includes icons for battery, plug, and caution.

Guidance and Manufacturer's Declaration

This blood pressure monitor is designed according to the European Standard EN1060. Non-invasive sphygmomanometers Part 1: General requirements and supplementary type requirements for electromagnetic compatibility measuring systems.

Maintenance

Protect your monitor from damage. Follow the directions below: Change or modifications not approved by the manufacturer will void your warranty.

Caution

DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

Storage

Keep your monitor in the storage case when not in use. Remove the arm cuff from the monitor.

Caution

To unplug the air plug, pull on the plastic air plug at the base of the tube, not the air plug itself.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

Specifications

Table with 2 columns: Product category, Electronic Sphygmomanometers. Lists model M2 (HEM-7143-E), LCD display, cuff pressure range, blood pressure measurement range, etc.

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DO NOT use this medidor in infants, toddlers, children or older adults. DO NOT adjust medication based on readings from this blood pressure monitor.

DO NOT use this medidor in areas containing high frequency radio waves. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

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DO NOT use this medidor in areas containing high frequency radio waves. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

Battery Handling and Usage

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

Battery Handling and Usage

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

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DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

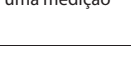
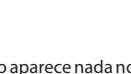
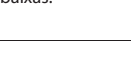
DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

DO NOT use this medidor on patients with atrial or ventricular fibrillation, a physician is qualified to diagnose and treat high blood pressure. DO NOT use this medidor in oxygen rich environments or near flammable gases.

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Automatic Upper Arm Blood Pressure Monitor M2 (HEM-7143-E)

All for Healthcare

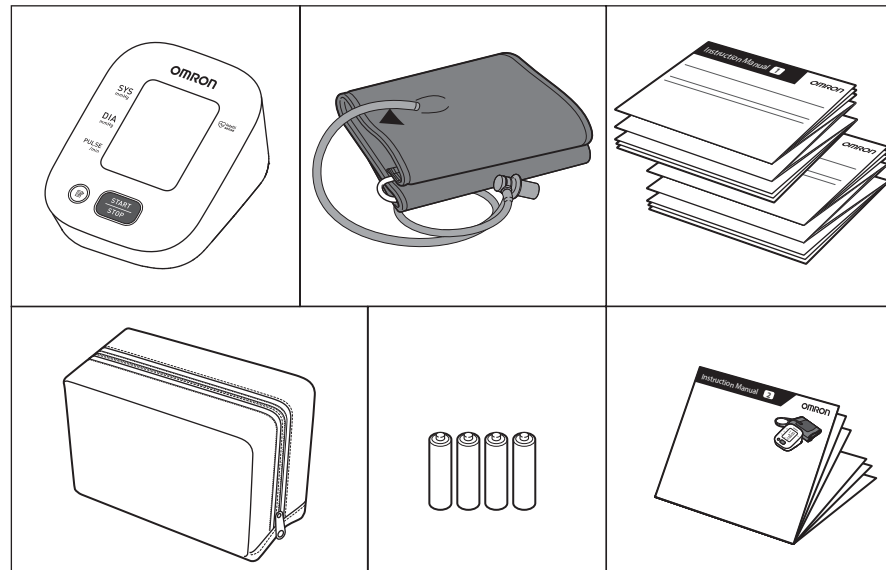
Read instruction manual ① and ② before use.

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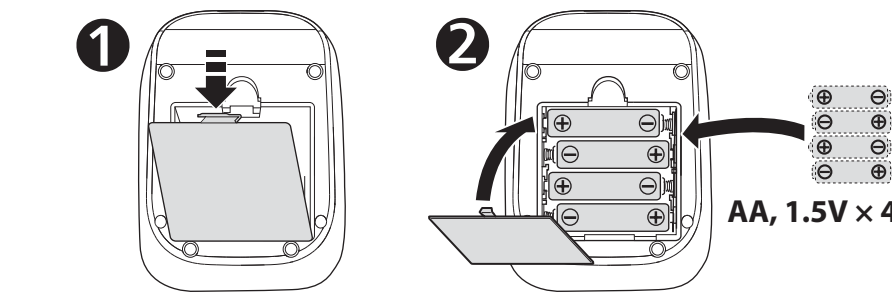
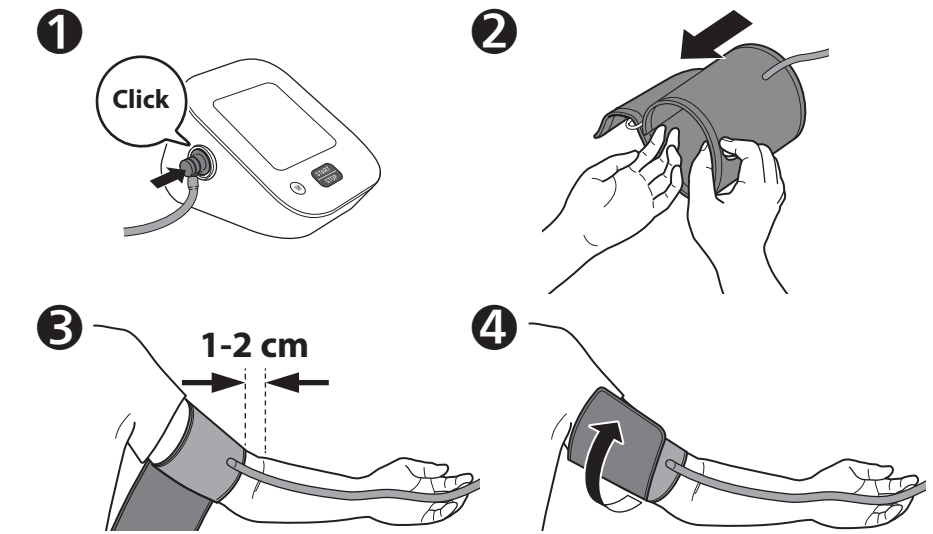
3 Tube side of the cuff should be 1 - 2 cm above the inside elbow.

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4 Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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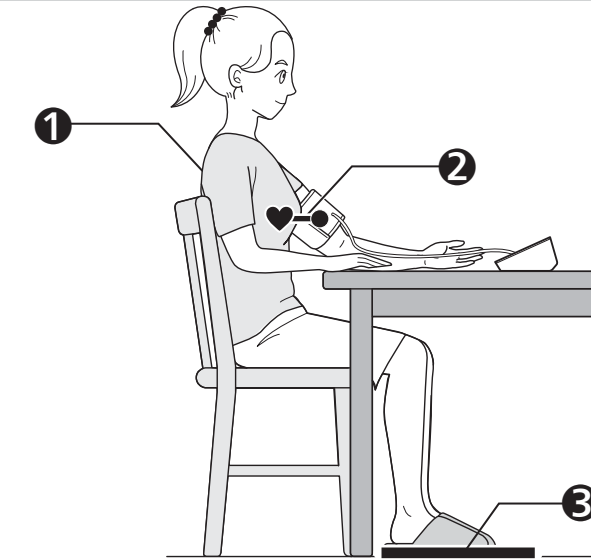
If taking measurements on the right arm, refer to:

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Instruction Manual ①
General Precautions

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1 Sit comfortably with your back and arm supported.

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2 Place the arm cuff at the same level as your heart.

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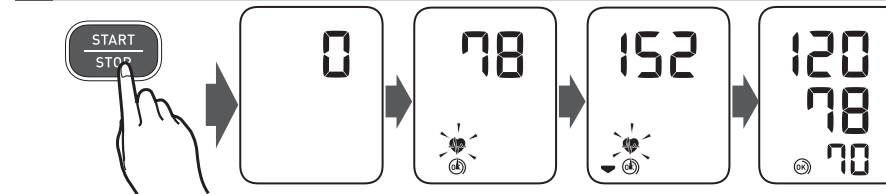
3 Keep feet flat, legs uncrossed, remain still and do not talk.

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When the [START/STOP] button is pressed, the measurement is taken and saved automatically.

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If your systolic pressure is more than 210 mmHg:

After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

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2 Appears when your body moves during a measurement. Remove the arm cuff, wait 2 - 3 minutes and try again.

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3 Appears when an irregular rhythm** is detected during a measurement. If it appears repeatedly, OMRON recommends to consult your physician.

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*The high blood pressure definition is based on the 2018 ESH/ESC Guidelines.

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**An irregular heart beat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected during a measurement.

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Error messages or other problems? Refer to:

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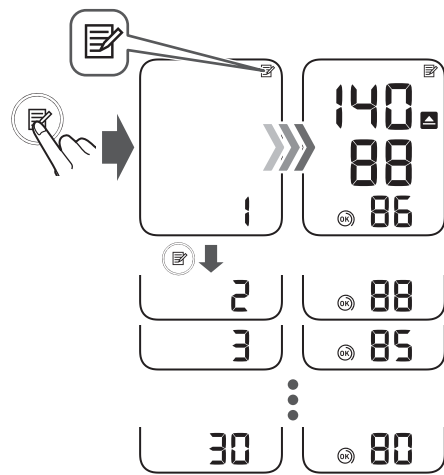
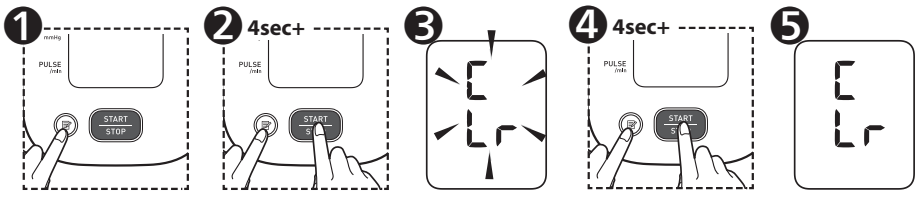
Do not throw the air plug away. The air plug can be applicable to the optional cuff.

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