# M7 Intelli IT AFib (HEM-7380T1-EBK)

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1.

## Applied part - Type BF Degree of protection against electric shock (leakage current)

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2.

## Class II equipment. Protection against electric shock

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3.

## Ingress protection degree provided by IEC 60529

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15.

**Identifier of cuffs compatible for the device**

Identificateur des brassards compatibles avec l'appareil  
Kennzeichnung der mit dem Gerät kompatiblen Manschetten  
Identifica i bracciali compatibili con il dispositivo  
Identificador para manguitos compatibles con el dispositivo  
Identificatie van manchetten die compatibel zijn met het apparaat  
Манжеты совместимые с устройством  
Cihaz ile uyumlu kolluکلarın tanıtım işareti  
علامات تحديد الشروط الضاغطة المتوافقة مع الجهاز

17.

**Not made with natural rubber latex**

Ne contient pas de latex de caoutchouc naturel  
Enthält kein Naturlatex  
Non contiene lattice di gomma naturale  
No contiene látex de caucho natural  
Bevat geen natuurrubberlatex  
Не содержит натуральный латекс  
Doğal kauçuk lateksten üretilmemiştir  
تم التصنيع دون استخدام لاتيكس المطاط الطبيعي

20.

**Need for the user to follow this instruction manual thoroughly for your safety. (Background: blue)**

L'utilisateur doit suivre attentivement ce mode d'emploi pour votre sécurité. (Fond : bleu)  
Damit die Sicherheit gewährleistet ist, muss der Benutzer diese Gebrauchsanweisung sorgfältig befolgen. (Hintergrund: blau)  
Per la propria sicurezza, l'utente deve seguire attentamente il presente manuale di istruzioni. (Sfondo: blu)  
Es necesario que el usuario siga rigurosamente este manual de instrucciones para su seguridad. (Fondo: azul)  
Noodzaak voor de gebruiker om zich voor de eigen veiligheid zorgvuldig aan deze gebruiksaanwijzing te houden (Achtergrond: blauw)  
Обратитесь к руководству по эксплуатации (Фон: синий)  
Güvenlik açısından kullanıcının bu kullanım kılavuzuna dikkatle uyması gerekir. (Arka plan: mavi)  
حاجة المستخدم إلى اتباع الإرشادات الواردة في دليل الإرشادات بالكامل للحفاظ على السلامة. (الخلفية: زرقاء)

16.

**Artery mark**

Marque d'artère  
Arterienmarkierung  
Contrassegno per l'arteria  
Marca de la arteria  
Adersymbol  
Маркер артерии  
Arter işareti  
علامة دالة على الشريان

19.

**Need for the user to consult this instruction manual**

L'utilisateur doit consulter le présent mode d'emploi  
Der Benutzer muss diese Gebrauchsanweisung lesen  
L'utente deve consultare il presente manuale di istruzioni  
Es necesario que el usuario consulte este manual de instrucciones  
Noodzaak voor de gebruiker om deze gebruiksaanwijzing te raadplegen  
Необходимо ознакомиться с данным руководством по эксплуатации  
Kullanıcının bu kullanım kılavuzuna başvurması gerekir  
حاجة المستخدم إلى الرجوع إلى المعلومات الواردة بتدليل الإرشادات هنا

21.

**Direct current**

Courant continu  
Gleichstrom  
Corrente diretta  
Corriente directa  
Gelijkstroom  
Постоянный ток  
Doğru akım  
التيار المباشر

22.

**Alternating current**

Courant alternatif  
Wechselstrom  
Corrente alternata  
Corriente alterna  
Wisselstroom  
Переменный ток  
Alternatif akım  
التيار المتناوب

23.

**Date of manufacture**

Date de fabrication  
Herstellungsdatum  
Data di fabbricazione  
Fecha de fabricación  
Productiedatum  
Дата изготовления в формате ГГГГММДД  
Üretim tarihi  
تاريخ التصنيع

24.

**Efficiency level of power supply**

Niveau d'efficacité de l'alimentation électrique  
Effizienz der Stromversorgung  
Livello di efficienza dell'alimentazione  
Grado de eficiencia de la alimentación  
Efficiëntieniveau van de voeding  
Уровень эффективности источника питания  
Güç kaynağının verimlilik seviyesi  
مستوى كفاءة مصدر الطاقة

25.

**Prohibited action**

Action interdite  
Verbotene Aktion  
Operazione proibita  
Acción prohibida  
Verboden handeling  
Запрещенное действие  
Yasaklanmış eylem  
إجراء محظور

26.

**To indicate generally elevated, potentially hazardous, levels of non-ionizing radiation, or to indicate equipment or systems. e.g. in the medical electrical area that include RF transmitters or that intentionally apply RF electromagnetic energy for diagnosis or treatment.**

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27.

**Recycle mark X: Material number Y: Material abbreviation Refer to 97/129/EC for more information.**

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**The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by OMRON HEALTHCARE Co., Ltd. is under license. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Other trademarks and trade names are those of their respective owners.**

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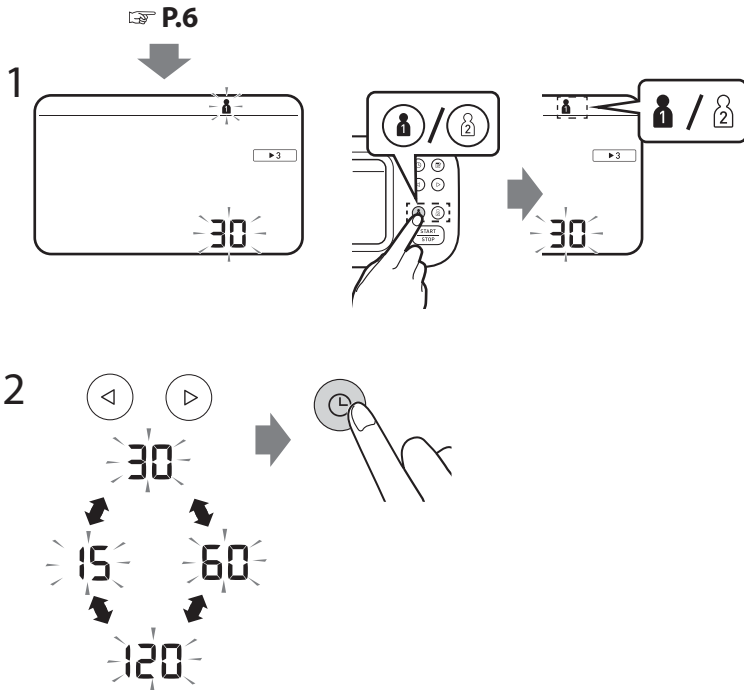
In the TruRead mode, your monitor automatically takes 3 consecutive readings at selected intervals and displays the average. The interval is set to 30 seconds by default. Refer to "Taking a measurement in TruRead mode" on page 15.

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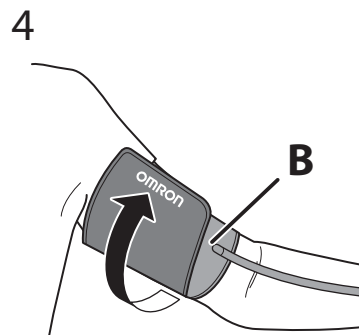
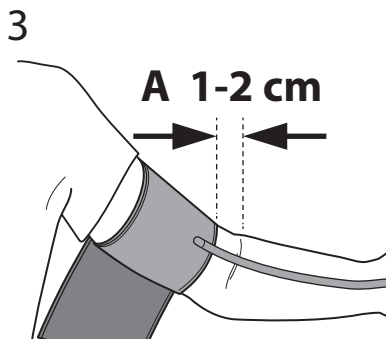
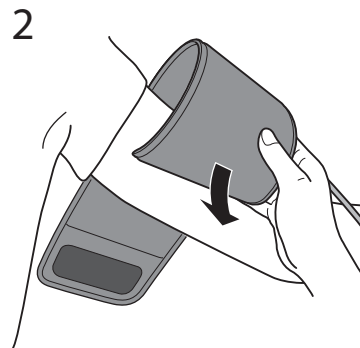
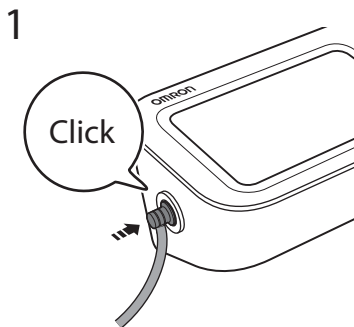
**A.** Tube side of the cuff should be 1 - 2 cm above the inside elbow.

**B.** Make sure that the air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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**If taking measurements on the right arm, refer to:**

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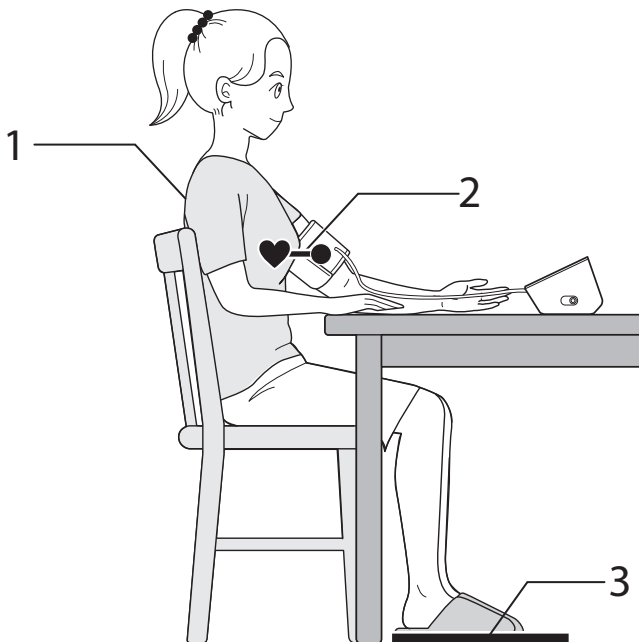
Relax and sit comfortably. Remain still and do not talk.

1. Keep your back and arm supported.
2. Keep the arm cuff at the same level as your heart.
3. Keep your feet flat and your legs uncrossed.

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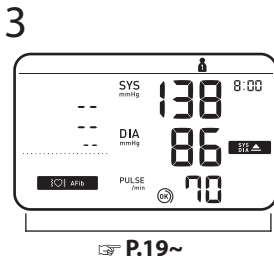
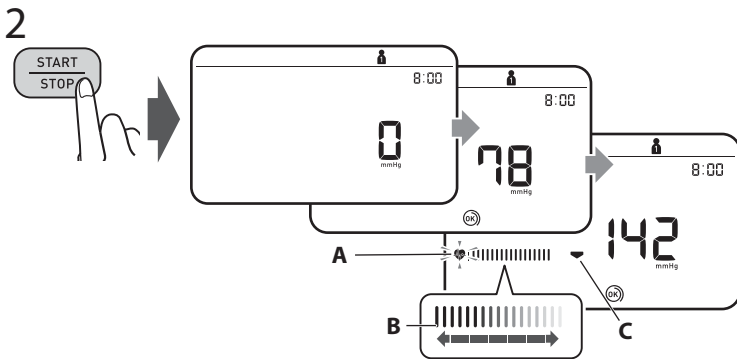
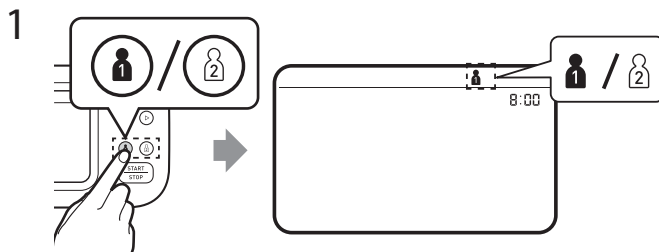
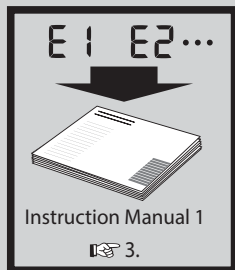
1. Select your user ID from buttons "1" or "2".
2. Press the [START/STOP] button.
  - A: Flashes at every heartbeat.
  - B: Moves along with the pulse strength while the cuff is deflating.
  - C: Appears while the cuff is deflating.
3. The reading is saved automatically.

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▶ 3

## Taking a measurement in TruRead mode

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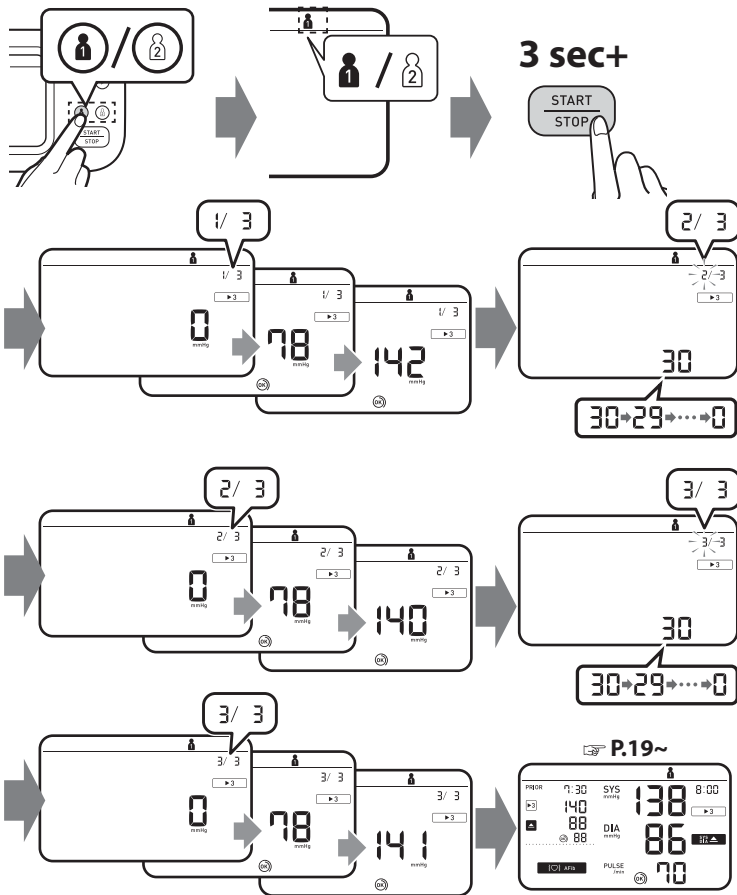
**In the TruRead mode, your monitor automatically takes 3 consecutive readings at selected intervals and displays the average. The interval is set to 30 seconds by default. Refer to "Setting the TruRead Mode Interval" on page 7.**

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
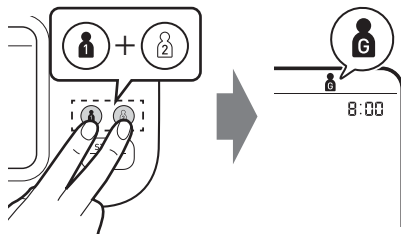
1

2

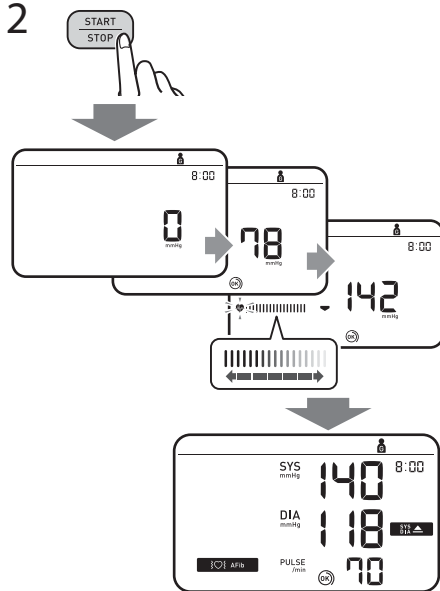**Takes a single measurement for another user. No readings are stored in the memory and TruRead mode is unavailable.**

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**If your systolic pressure is more than 210 mmHg:  
After the arm cuff starts to inflate, press and hold the [START/STOP]  
button until the monitor inflates 30 to 40 mmHg higher than your  
expected systolic pressure. Do not inflate above 299 mmHg.**

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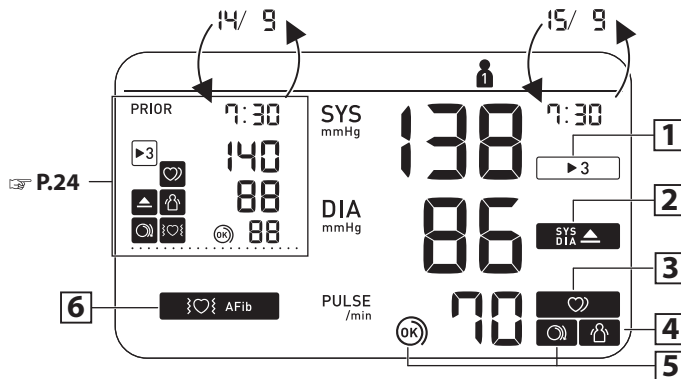
1

**Appears when the reading was taken in TruRead mode.**

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2

**Appears if "SYS" is 135 mmHg or above and/or "DIA" is 85 mmHg\* or above.**

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SYS  
DIA

3

**Appears when an irregular rhythm\*\* is detected during a measurement. If it continues to appear, consulting with your physician is recommended.**

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**4**

**Appears when your body moves during a measurement\*\*\*. Remove the arm cuff, wait 2-3 minutes and try again.**

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**Appears if a possibility of AFib was detected during a measurement. This is not a diagnosis, it is only a potential finding for AFib. You should contact your physician to discuss the findings.**

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E1  
E2  
•  
•

#### Error messages or other problems? Refer to:

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**Instruction Manual 1**

**3.**

**\* The high blood pressure definition is based on the 2021 ESH/ESC Guidelines.**

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**\*\* An irregular heartbeat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected while your monitor is measuring blood pressure.**

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\*\*\* The body movement function is disabled when a possibility of AFib or irregular heartbeat is detected during a measurement.

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## Individual TruRead Readings in Memory

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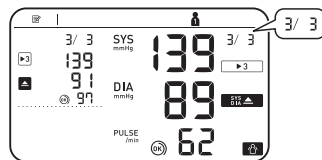
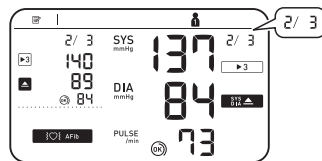
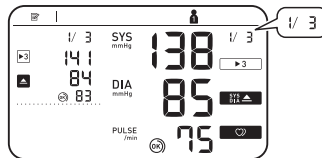
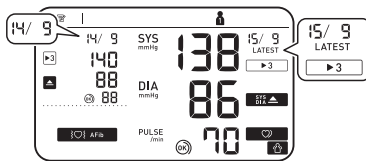
**While viewing the reading taken in TruRead mode, press  to view the individual readings. Individual TruRead readings will appear with  3 symbol on the left for the past, and on the right for the latest readings.**

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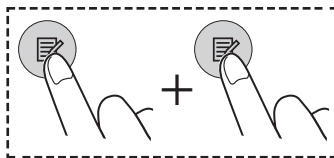
 **AVG Morning/Evening**  
 **AVG Weekly Averages**

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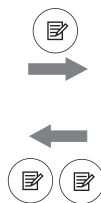
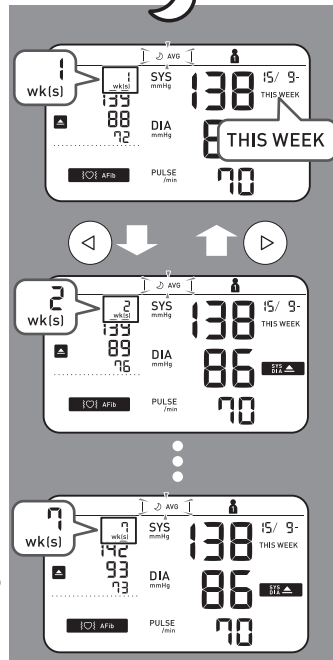
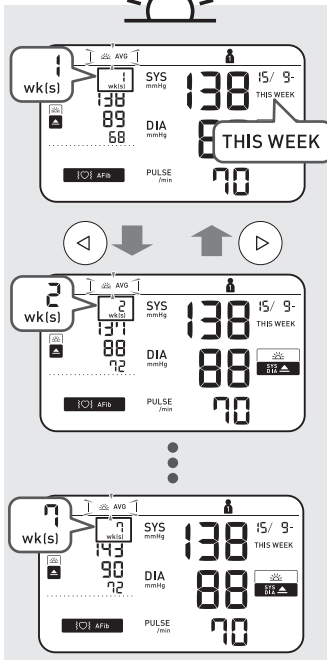
2 wks. ago

7 wks. ago

**Appears if "SYS" is 135 mmHg or above and/or "DIA" is 85 mmHg or above in the morning weekly average.**

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**To know how to calculate weekly averages, refer to section 10 of the Instruction Manual 1.**

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AVG

## Average of the Latest 2 or 3 Readings Taken within a 10 Minute Span

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