Long Life Pads

HV-LLPAD-E Instruction Manual

1615604-8F

Manufacturer information table including OMRON HEALTHCARE Co., Ltd. details in English and Arabic.

EU representative information table listing OMRON HEALTHCARE EUROPE B.V. and its address in the Netherlands.

Production facility information table for Sekisui Kasei Kanto Co., Ltd. in Japan.

Subsidiaries and distributors table listing various regional agents and distributors worldwide.

Origin information table stating 'Made in Japan' and 'Hergesteld in Japan'.

Description of symbols table mapping icons to manual references in various languages.

Temperature limitation table with icons and text in multiple languages.

Humidity limitation table with icons and text in multiple languages.

Atmospheric pressure limitation table with icons and text in multiple languages.

Proper disposal of product table with icons and text in multiple languages.

CE marking table with icons and text in multiple languages.

MD marking table with icons and text in multiple languages.

Manufacturer date table with icons and text in multiple languages.

LOT marking table with icons and text in multiple languages.

UKCA marking table with icons and text in multiple languages.

UDI marking table with icons and text in multiple languages.

Use-by-date table with icons and text in multiple languages.

Issue date and publication information table.

Intended Use

The Long Life Pads should be used only for applicable Omron devices and applied on normal, healthy, dry, clean skin of adults only.

Important safety information

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

NEVER APPLY THE PADS TO THESE BODY AREAS:

Diagram showing prohibited application areas with icons and text in English and Arabic.

- Apply pads to normal, healthy, dry, and clean skin (of adult patients) because it may otherwise disrupt the healing process.

Caution Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

- Detach the pads before replacing the batteries.

Precautions

- To avoid moving the pads to another location while the unit is on. Therapy won't work with just 1 pad. You must use 2 pads at the same time.

How to Use

Apply the pads to the body as described in your OMRON TENS device instruction manual.

- Snap either electrode cord of your OMRON TENS device to each of the pads.

Cleaning and storage

The unit is designed for repeated use over time. The pads will last up to 150 uses, or 5 months (based on use 1/day). Here are important cleaning and storage instructions:

Cleaning the pads

- Turn your OMRON TENS device off and remove the electrode cord from the pads.

Storing the pads

- Turn your OMRON TENS device off and remove the electrode cord from the pads.

Specifications

Product specifications table including product category, model, operating conditions, and weight.

Notes: These specifications are subject to change without notice. Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life.

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Diagram showing prohibited application areas with icons and text in German and Arabic.

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