
Probe Cover MC-EP2 (MC-EP2-E)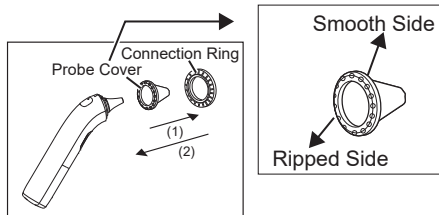

For Gentle Temp 520 (MC-520-E) Gentle Temp 521 (MC-521-E)

QG-MC-EP2-E-Gr.1-05-11/2022

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→ How to install a new Probe Cover. For MC-520, MC-521

EN

Always use a clean, undamaged and new probe cover. ⚠
Ensure that the ear canal is clean.

(1) Attach the probe cover to the connection ring.

- ** Make sure to place the connection ring over the smooth side of the probe cover as shown in the drawing.**
- (2) Align the center of probe cover with the center of probe.
 - (3) Push the connection ring until you hear a click. This means that the probe cover is tightly secured.
- The connection ring will only be used as an aid for applying the probe cover to the ear thermometer tip.

Note: Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

For the safe and effective use of this product you should always follow the guidance of the OMRON ear thermometer instruction manual.

Warning: Keep the probe covers and connection ring away from children.

Do not use one probe cover more than once. This can lead to cross infections such as external otitis.

Description of symbols that, depending on a model, can be found on the product itself, product sales package or IM.			
	CE Marking		Temperature limitation
	Need for the user to consult the instructions for use		Humidity limitation
	Do not re-use		Atmospheric pressure limitation
	LOT number		Medical Device
	UKCA Marking		Unique device identifier
Product production date is integrated in a LOT number, which is placed on the sales package: the first 4 digits mean year of production, the next 2 digits mean month of production, the next 2 digits mean day of the production.			

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