**OMRON**

**Automatic Wrist Blood Pressure Monitor**

**RS4 (HEM-6181-E)**

**Instruction Manual ①**

IM1-HEM-6181-E-05-01/2022  
Issue Date: 2022-06-27  
5681802-7b

**All for Healthcare**


Read the instruction manual before use.

Important safety information is in Instruction Manual ①.

How to set up the wrist monitor is in Instruction Manual ②.


**1. Know Your Monitor**

- 1.1 Contents
- 2 "AAA" alkaline batteries, storage case, instruction manual 1 and 2
- 1.2 Monitor

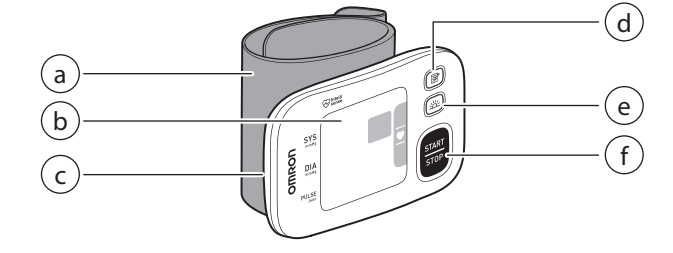- a. Wrist cuff (Wrist circumference 13.5 cm to 21.5 cm)
- b. Display
- c. Battery compartment
- d. (Memory) button
- e. (Morning average) button
- f. (START/STOP) button

**1.3 Display and Symbols**
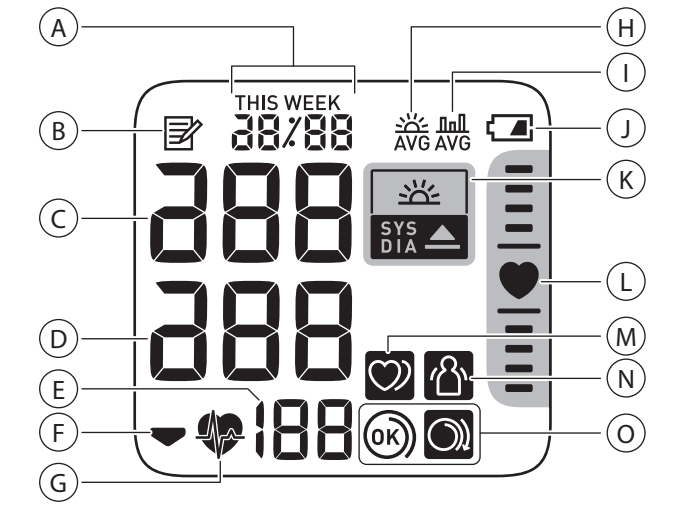
<b>A</b>	<b>Date and time display</b>
<b>B</b>	<b>Memory symbol</b> Appears when viewing readings stored in the memory.
<b>C</b>	<b>Systolic blood pressure reading</b>
<b>D</b>	<b>Diastolic blood pressure reading</b>
<b>E</b>	<b>Pulse display / Memory number</b> Pulse rate appears after the measurement. When pressing the <b>OK</b> button, the memory number appears for approximately one second before the pulse rate appears on the display.
<b>F</b>	<b>Deflation symbol</b> Appears during cuff deflation.
<b>G</b>	<b>Heartbeat symbol</b> Flashes while taking a measurement.
<b>H</b>	<b>Morning average symbol</b> Appears when viewing the weekly average readings taken in the morning. Weekly averages for the measurements taken in the morning within 4 weeks appear on the display.
<b>I</b>	<b>Average value symbol</b> Appears when viewing the average readings of the latest 2 or 3 readings taken within 10 minutes span.
<b>J</b>	<b>Battery symbol (low)</b> Flashes when batteries are low.
<b>K</b>	<b>Battery symbol (depleted)</b> Appears when batteries are depleted.
<b>L</b>	<b>Hypertension symbol</b> Appears if your systolic blood pressure is 135 mmHg or above and/or the diastolic blood pressure is 85 mmHg or above.
<b>M</b>	<b>Morning hypertension symbol</b> Appears when the morning weekly average is 135/85 mmHg or above.
<b>N</b>	<b>Positioning indicator</b> Your monitor has a built-in advanced positioning sensor that is used as an aid in determining if the monitor is at the correct height. The "♥" symbol appears when your monitor is at the correct position relative to your heart during a measurement. When your monitor is higher or lower position relative to your heart, the heart position locator bar will appear up above, or below the heart symbol to guide correct wrist positioning.
<b>O</b>	<b>Irregular heartbeat symbol</b> Appears along with readings when an irregular rhythm is detected 2 or more times during a measurement. An irregular heartbeat rhythm is defined as a rhythm that is 25% less or 25% more than the average rhythm detected while your monitor is measuring blood pressure. If it continues to appear, we recommend you to consult with and follow the directions of your physician.
	<b>Movement error symbol</b> Appears along with readings when your body is moving during a measurement. If it appears, remove the wrist cuff, and wait for 2 to 3 minutes. Then, remain still and take another measurement.
	<b>Cuff wrap guide symbol (OK)</b> Appears if the wrist cuff is wrapped around the wrist correctly while taking a measurement. It also appears when viewing past readings.
	<b>Cuff wrap guide symbol (loose)</b> Appears if the wrist cuff is not wrapped around the wrist correctly while taking a measurement. It also appears when viewing past readings.

**2013 ESH/ESC\* Guidelines for the management of arterial hypertension**

	<b>Office</b>	<b>Home</b>
Systolic Blood Pressure	≥ 140 mmHg	≥ 135 mmHg
Diastolic Blood Pressure	≥ 90 mmHg	≥ 85 mmHg


\* European Society of Hypertension (ESH) and European Society of Cardiology (ESC).  
NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.

**2. Using Memory Function**

**2.1 Viewing the Average Reading**  
Your monitor can calculate an average reading based on the latest 2 or 3 readings taken within 10 minutes span.

1. When your monitor is off, press and hold the **OK** button for more than 2 seconds.


**Note**  
There are only 2 readings in the memory for that period, the average will be based on these 2 readings.  
If the average reading is high (refer to section 1.3), the "H" symbol appears.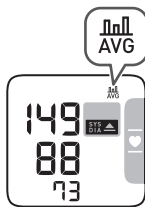


**2.2 Viewing Morning Week Averages**

Your monitor calculates and displays weekly averages for the measurements taken in the morning for 4 weeks.

1. Press the **OK** button once.  
The morning weekly average for the current week "THIS WEEK" appears on the display.  
If the morning weekly average is high (refer to section 1.3), the "H" symbol appears.
2. Press the **OK** button repeatedly to display the average for the previous weeks.  
Your monitor displays "1 WEEK" for the previous week's average, to "3 WEEK" for the oldest set of averages.
3. Press the (START/STOP) button to turn off your monitor.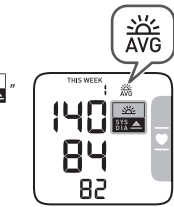


**Morning Weekly Average Calculation**  
This is the average for the measurements taken during the morning (4:00 - 9:59) between Sunday and Saturday. Only the first 3 measurements taken in the morning between 4:00 - 9:59 will be used to calculate the morning average for each day.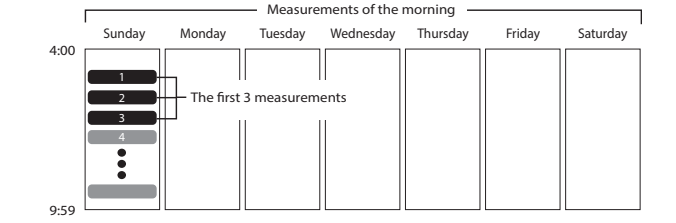

**3. Other Settings**

**3.1 Disabling/Enabling Positioning Indicator**  
The positioning indicator is enabled as default.

1. When your monitor is off, press and hold the **OK** button for more than 10 seconds.

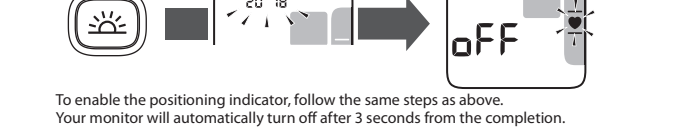**4. Error Messages and Troubleshooting**

In case of any of the below problems occur during measurement, first check that no other electrical device is within 30 cm. If the problem persists, please refer to the table below.

Display/Problem	Possible Cause	Solution
E1	The wrist cuff is not applied correctly.	Apply the wrist cuff correctly, then take another measurement. Refer to section 3 in instruction manual 2.
E3	Air is leaking from the wrist cuff.	Contact your OMRON retail outlet or distributor.
E4	The wrist cuff is overinflated exceeding 300 mmHg.	Do not touch the wrist cuff while taking a measurement.
E5	You move or talk during a measurement. Vibrations disrupt a measurement.	Remain still and do not talk during a measurement.
E6	The pulse rate is not detected correctly.	Remain still and sit correctly during a measurement. If the "♥" symbol continues to appear, we recommend you to consult with your physician.
E7	Your wrist is moved up and down during a measurement.	Do not move your wrist, then take another measurement.
E8	The monitor is malfunctioned.	Press the (START/STOP) button again. If "E" still appears, contact your OMRON retail outlet or distributor.
Heart symbol	The positioning indicator is disabled.	Enable the positioning indicator. Refer to section 3 in this manual.
Batteries are low.	Batteries are completely depleted.	Replacing all batteries with 2 new alkaline batteries is recommended. Refer to section 1 in instruction manual 2.
Battery polaries are not properly aligned.	Battery polarities are not properly aligned.	Check the battery installation for proper placement. Refer to section 1 in instruction manual 2.
Blood pressure varies constantly. May factors including stress, time of day, and/or how you apply the wrist cuff, may affect your blood pressure.	Blood pressure varies constantly. May factors including stress, time of day, and/or how you apply the wrist cuff, may affect your blood pressure.	Review section 9 in this manual and section 3 - 5 in instruction manual 2.
Readings appear too high or too low.	Batteries are depleted.	Immediately replace all batteries with 2 new alkaline batteries. Refer to section 1 in instruction manual 2.
Any other problems occur.	Batteries are completely depleted.	Immediately replace all batteries with 2 new alkaline batteries. Refer to section 1 in instruction manual 2.

**5. Maintenance**

**5.1 Maintenance**  
To protect your monitor from damage, please follow the directions below:  
- Changes or modifications not approved by the manufacturer will void the user warranty.

- Caution**
  - DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

**5.2 Storage**  
Keep your monitor in the storage case when not in use.  
- Store your monitor in a clean, safe location.  
- If your monitor is wet:  
- In locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.

**5.3 Cleaning**  
Do not use any abrasive or volatile cleaners.  
- Use a soft dry cloth or a soft cloth moistened with neutral soap to clean your monitor and wrist cuff, and then wipe them with a dry cloth.  
- Do not wash or immerse your monitor and wrist cuff in water.  
- Do not use gasoline, thinner or similar solvents to clean your monitor and wrist cuff.

- 5.4 Calibration and Service**
  - The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
  - It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON Customer Service at the address given on the packaging or attached literature.

**6. Specifications**

Product category	Electronic Sphygmomanometers
Product description	Automatic Wrist Blood Pressure Monitor
Model (Code)	RS4 (HEM-6181-E)
Display	LCD digital display
Cuff pressure range	0 to 299 mmHg
Blood pressure measurement range	SYS: 60 to 260 mmHg DIA: 40 to 215 mmHg
Pulse measurement range	40 to 180 beats / min.
Accuracy	Pressure: ±3 mmHg Pulse: ±5% of display reading
Inflation	Automatic by electric pump
Deflation	Automatic rapid deflation
Measurement method	Oscillometric method
Operation mode	Continuous operation
IP classification	IP 22
Rating	DCV 3.0 W
Power source	2 "AAA" alkaline batteries 1.5V
Battery life	Approximately 300 measurements (using new alkaline batteries)
Durable period (Service life)	5 years
Operating conditions	+10°C to +40°C / 15 to 90% RH (non-condensing) / 800 to 1060 hPa
Storage / Transport conditions	-20°C to +60°C / 10 to 90% RH (non-condensing)
Weight	Approximately 86 g not including batteries
Dimensions	Approximately 93 mm (w) × 62 mm (h) × 20 mm (l) (not including the wrist cuff)
Measurable wrist circumference	13.5 to 21.5 cm
Memory	Stores up to 60 readings
Contents	Monitor, storage case, 2 "AAA" alkaline batteries, instruction manual 1 and 2
Protection against electric shock	Internally powered ME equipment
Applied part	Type BF wrist cuff
Maximum temperature of the applied part	Lower than +48°C

**Note**  
- These specifications are subject to change without notice.  
- In the clinical validation study, K5 was used on 85 subjects for determination of diastolic blood pressure.  
- This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2014 and EN ISO 81060-2:2019 - A1:2020.  
- IP classification is degree of protection provided by enclosures in accordance with IEC 60529. This monitor is protected against solid foreign objects of 12.5 mm diameter and greater such as a finger, and against oblique falling water drops which may cause issues during a normal operation.  
- This monitor has not been validated for use on pregnant patients.  
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

**7. Limited Warranty**

Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing. It is designed to give you every satisfaction, provided that it is properly operated and maintained as described in the instruction manual. This product is warranted by OMRON for a period of 5 years after the date of purchase. The proper construction, workmanship and materials of this product are guaranteed by OMRON. During this period of warranty OMRON will, without charge for labour or parts, repair or replace the defect product or any defective parts. The warranty does not cover any of the following:  
A. Transport costs and risks of transport.  
B. Costs for repairs and / or defects resulting from repairs done by unauthorised persons.  
C. Periodic check-ups and maintenance.  
D. Failure or wear of optional parts or other attachments other than the main device itself, unless explicitly warranted above.  
E. Costs arising due to non-acceptance of a claim (those will be charged for).  
F. Damages of any kind including personal caused accidentally or from misuse.  
G. Calibration service is not included within the warranty.  
Should warranty service be required please apply to the dealer whom the product was purchased from or an authorised OMRON distributor. For the address refer to the product packaging / literature or to your specialised retailer. If you have difficulties in finding OMRON customer services, contact us for information.  
www.omron-healthcare.com  
Repair or replacement under the warranty does not give rise to any extension or renewal of the warranty period.  
The warranty will be granted only if the complete product is returned together with the original invoice / cash ticket issued to the consumer by the retailer.

**8. Guidance and Manufacturer's Declaration**

This blood pressure monitor is designed according to the European Standard EN 1060, Non-Invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.  
This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., Japan. The Core component for OMRON blood pressure monitors, which is the Pressure Sensor, is produced in Japan.

**Important information regarding Electro Magnetic Compatibility (EMC)**  
HEM-6181-E manufactured by OMRON HEALTHCARE Co., Ltd. conforms to EN60601-1-2:2015 Electro Magnetic Compatibility (EMC) standard.  
Further information in accordance with this EMC standard is available at www.omron-healthcare.com. Refer to the EMC information for HEM-6181-E on the website.

**Correct Disposal of This Product (Waste Electrical & Electronic Equipment)**

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact their supplier or the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

**9. Blood Pressure Measurement Tips**

To help ensure an accurate reading, follow these directions:  
- Stress raises blood pressure. Avoid taking measurements during stressful times.  
- Measurements should be taken in a quiet place.  
- It is important to take measurements at some times each day. Taking measurements in the morning and in the evening is recommended.  
- Remember to have a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure.  
Please use the Blood Pressure Diary to keep records of several readings over a certain period of time. To download PDF files of the diary, visit www.omron-healthcare.com.

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