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The **Bluetooth®** word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by OMRON HEALTHCARE Co., Ltd. is under license. Other trademarks and trade names are those of their respective owners. App Store and App Store logo are service marks of Apple Inc., registered in the U.S and other countries. Google Play and Google Play logo are trademarks of Google LLC.

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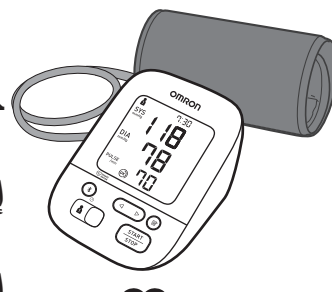
Intellisense

IM2-HEM-7155T-E-05-01/2022
2870441-5D

Read instruction manual ① and ② before use.

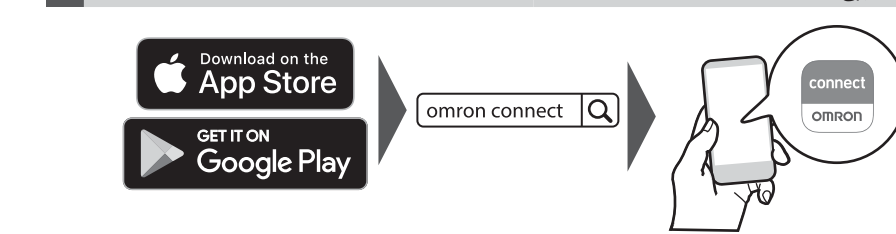
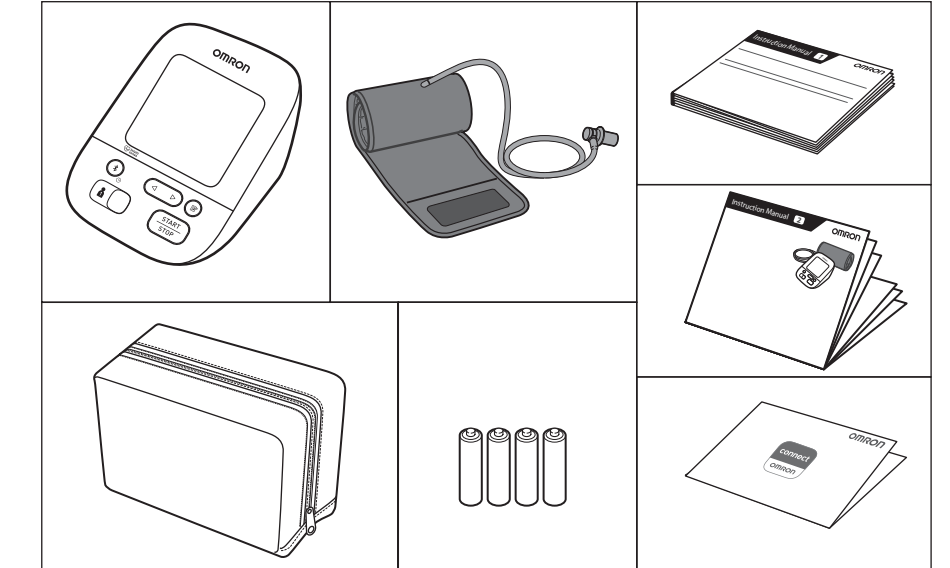
- FR Lire le mode d'emploi ① et ② avant l'utilisation.
- DE Lesen Sie vor der Verwendung die Gebrauchsanweisung ① und ②.
- IT Leggere il manuale di istruzioni ① e ② prima dell'uso.
- ES Lea el manual de instrucciones ① y ② antes del uso.
- NL Lees de gebruiksaanwijzing ① en ② voor gebruik.
- RU Прочтите руководство по эксплуатации ① и ② перед использованием.
- TR Kullanmadan önce, kullanım kılavuzu ① ve ②'yi okuyun.

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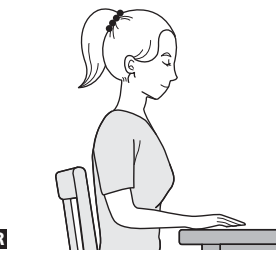
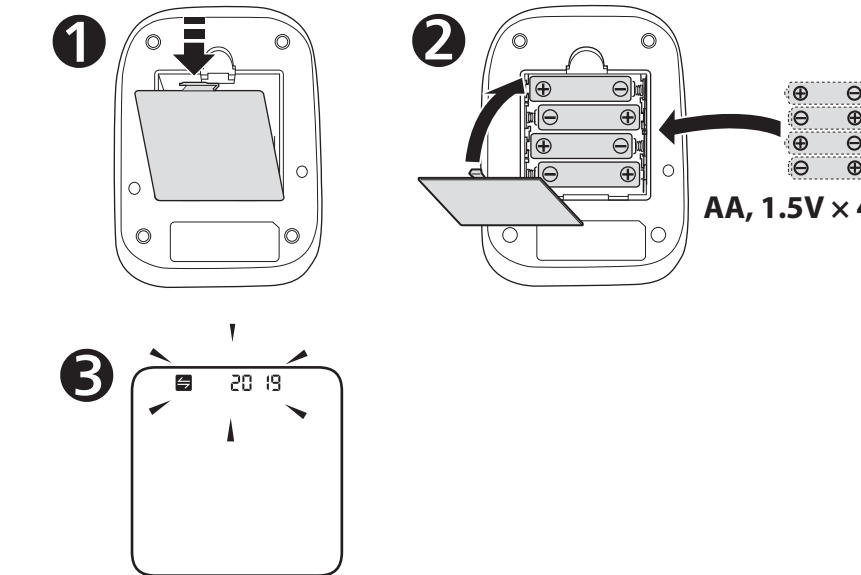
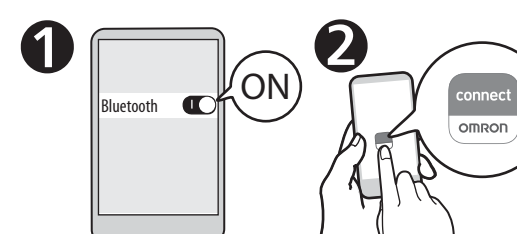
The date and time will automatically be set when your monitor is paired with the app.

- FR La date et l'heure seront automatiquement réglées lorsque votre tensiomètre sera jumelé avec l'application.
- DE Wenn Ihr Messgerät mit der App verknüpft ist, werden Datum und Uhrzeit automatisch eingestellt.
- IT La data e l'ora verranno impostate automaticamente una volta che il misuratore verrà associato alla app.
- ES La fecha y la hora se ajustarán automáticamente cuando el monitor se sincronice con la aplicación.
- NL De datum en tijd worden automatisch ingesteld wanneer uw bloeddrukmeter is gekoppeld aan de app.
- RU Дата и время установятся автоматически после синхронизации с приложением.
- TR Ölçüm cihazınız uygulamayla eşleştirildiğinde tarih ve saat otomatik olarak ayarlanır.

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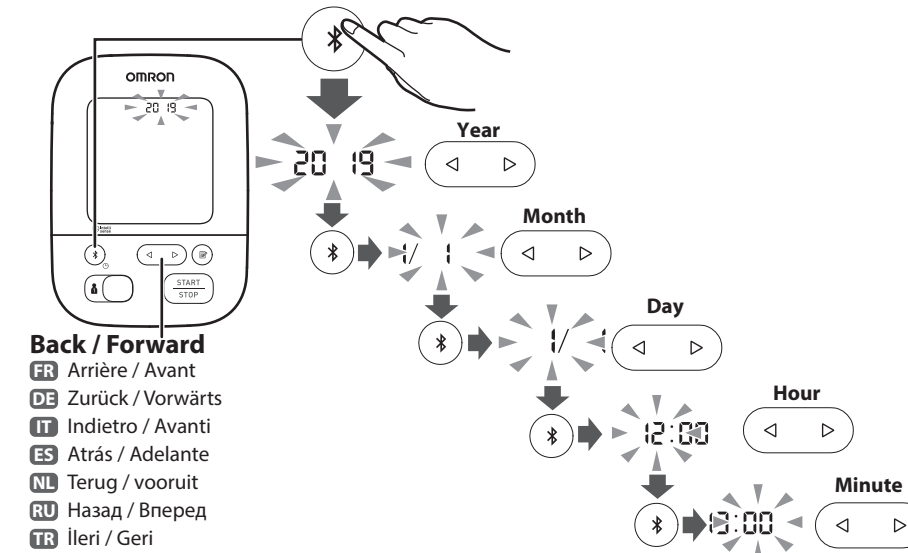
If your monitor is paired with your smart device, date and time is set automatically. When you need to set them manually, set year > month > day > hour > minute.

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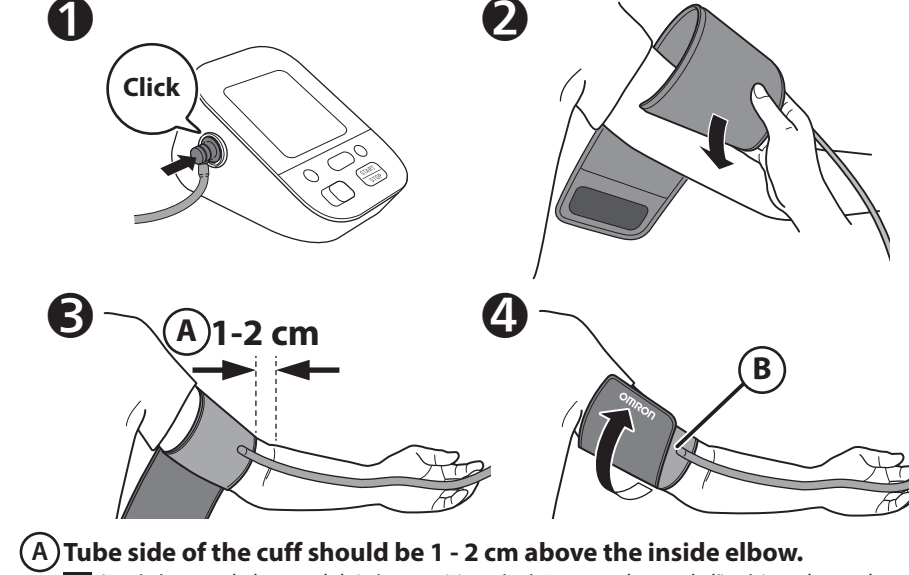
AR Tube side of the cuff should be 1 - 2 cm above the inside elbow.

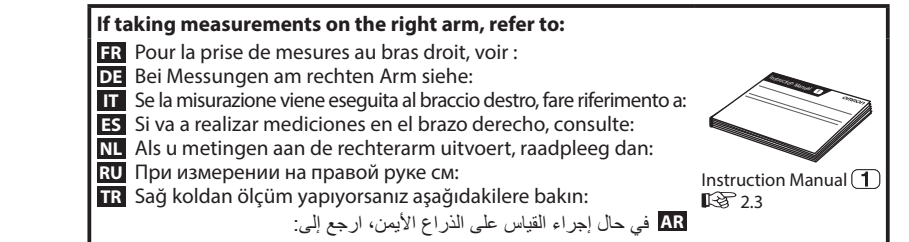
- FR Le côté tuyau du brassard doit être positionné 1 à 2 cm au-dessus de l'intérieur du coude.
- DE Das Manschettenstück mit dem Schlauch muss 1 bis 2 cm oberhalb des Ellbogens liegen.
- IT Il lato del bracciale con il tubo deve trovarsi al di sopra dell'interno del gomito, a una distanza di circa 1 o 2 cm.
- ES El lado del tubo del manguito deberá quedar 1 o 2 cm por encima de la parte interna del codo.
- NL De kant met de slang van de manchet moet 1 - 2 cm boven de binnenkant van de elleboog liggen.
- RU Край манжеты с трубкой должен находиться на 1-2 см выше локтевого сгиба.
- TR Kolluğun boru tarafı, dirsek içinin 1 ila 2 cm üstünde olmalıdır.

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Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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
AR If taking measurements on the right arm, refer to:

- FR Pour la prise de mesures au bras droit, voir :
- DE Bei Messungen am rechten Arm siehe:
- IT Se la misurazione viene eseguita al braccio destro, fare riferimento a:
- ES Si va a realizar mediciones en el brazo derecho, consulte:
- NL Als u metingen aan de rechterarm uitvoert, raadpleeg dan:
- RU При измерении на правой руке см:
- TR Sağ koldan ölçüm yapıyorsanız aşağıdakilere bakın:

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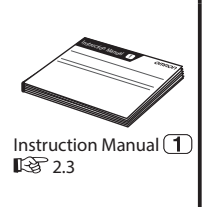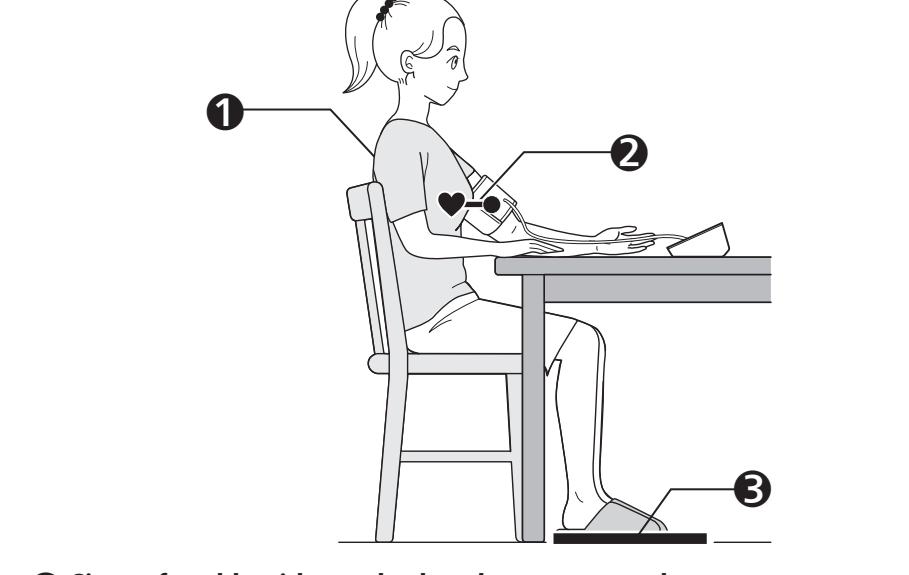
Sit comfortably with your back and arm supported.

- FR S'asseoir confortablement de manière à ce que le dos et le bras soient bien soutenus.
- DE Bequem hinsetzen, mit dem Rücken anlehnen und den Arm auf eine Unterlage legen.
- IT Sedere comodamente con la schiena e il braccio ben sostenuti.
- ES Siéntese cómodamente de modo que la espalda y el brazo estén bien apoyados.
- NL Neem een comfortabele zitpositie in, met uw rug en arm ondersteund.
- RU Сядьте удобно, чтобы спина и рука опирались на что-либо.
- TR Sirtınız ve kolunuz desteklenerek rahat bir şekilde oturun.

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Place the arm cuff at the same level as your heart.

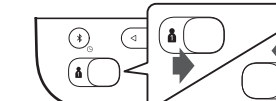
- FR Le brassard doit se trouver au même niveau que votre cœur.
- DE Die Manschette auf Herzhöhe platzieren.
- IT Posizionare il bracciale allo stesso livello del cuore.
- ES Coloque el manguito al mismo nivel que el corazón.
- NL Plaats de armmanchet op hetzelfde niveau als uw hart.
- RU Манжета должна находиться на уровне сердца.
- TR Kolluğu kalbinizle aynı düzeye getirin.

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Keep feet flat, legs uncrossed, remain still and do not talk.

- FR Garder les pieds à plat, les jambes non croisées, ne pas bouger et ne pas parler.
- DE Füße flach auf den Boden stellen, die Beine nicht kreuzen, still halten und nicht sprechen.
- IT Tenere i piedi ben poggiati, non incrociare le gambe, rimanere fermi e non parlare.
- ES Mantenga los pies planos, las piernas sin cruzar, quédese quieto y no hable.
- NL Houd de voeten plat, benen niet gekruist, zit stil en praat niet.
- RU Поставьте ноги ровно, не перекрещивая их, не двигайтесь и не разговаривайте.
- TR Ayaklarınızı düz, bacaklarınızı açık tutun, hareketsiz kalın ve konuşmayın.

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
Switching user ID enables you to save readings for 2 people.

- FR Changer d'ID Utilisateur permet d'enregistrer les résultats pour 2 personnes.
- DE Mit verschiedenen Benutzer-IDs lassen sich Messwerte für 2 Personen speichern.
- IT Cambiare ID utente permette di registrare i risultati relativi a 2 persone.
- ES Cambiar de ID de usuario le permite guardar las lecturas de 2 personas.
- NL Door het wisselen van gebruikers-ID kunt u metingen voor 2 personen opslaan.
- RU Переключение идентификатора пользователя позволяет сохранять результаты для 2-x людей.
- TR Kullanıcı kimlikleri arasında geçiş yapma, 2 kişi için ölçüm değerlerini kaydetmenizi sağlar.

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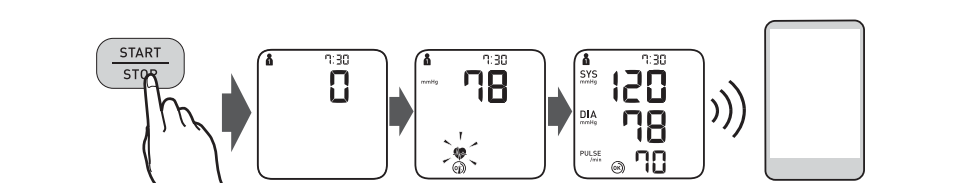When the [START/STOP] button is pressed, the measurement is taken and saved automatically. Open the app to transfer the reading.

FR Après une pression sur le bouton [START/STOP], la mesure démarre et le résultat est enregistré automatiquement. Ouvrir l'application pour transférer le résultat.
DE Beim Drücken der Taste [START/STOP] erfolgt die Messung. Die Messwerte werden automatisch gespeichert. Öffnen Sie die App, um die Messwerte zu übertragen.
IT Quando si preme il pulsante [START/STOP], la misurazione viene eseguita e salvata automaticamente. Aprire la app per trasferire i risultati.
ES Al pulsar el botón [START/STOP], se obtiene la medición, que se guarda automáticamente. Abra la aplicación para transferir la lectura.
NL Wanneer op de knop [START/STOP] wordt gedrukt, wordt de meting automatisch uitgevoerd en opgeslagen. Open de app om de meting te versturen.
RU После того, как кнопка [START/STOP] нажата, измерение выполняется и сохраняется автоматически. Откройте приложение, чтобы передать результаты.
TR [START/STOP] düğmesine basıldığında ölçüm yapılır ve otomatik olarak kaydedilir. Ölçüm değerini aktarmak için uygulamayı açın.

The guest mode can be used to take a single measurement for another user. No readings are stored in the memory.

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If your systolic pressure is more than 210 mmHg:

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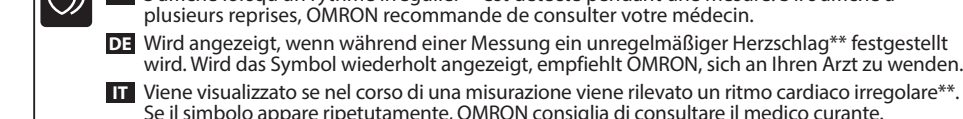
12.2 Average of the Latest 2 or 3 Readings Taken within a 10 Minute Span

FR Moyenne des 2 ou 3 dernières mesures prises en l'espace de 10 minutes
DE Mittelwert der letzten 2 oder 3 Messwerte, die in einem 10-Minuten-Zeitraum erfasst wurden
IT Media degli ultimi 2 o 3 risultati ottenuti nell'arco di 10 minuti
ES Promedio de las 2 o 3 últimas lecturas realizadas en un intervalo de 10 minutos
NL Gemiddelde van de laatste 2 of 3 metingen uitgevoerd binnen een tijdsruimte van 10 minuten
RU Среднее значение из последних 2-x или 3-x результатов полученных в течение 10 минут
TR 10 Dakika Aralıkta Alınan Son 2 veya 3 Ölçüm Değerinin Ortalaması
AR متوسط آخر قراءتين أو 3 قراءات تم الحصول عليها في مدة زمنية تبلغ ١٠ دقائق

12.3 Deleting All Readings

FR Suppression de toutes les mesures
DE Löschen aller Messwerte
IT Cancellazione di tutti i risultati
ES Eliminación de todas las lecturas
NL Alle metingen verwijderen
AR حذف جميع القراءات

1 2 3 4 5
FR Messages d'erreur ou autres problèmes? Refer to:
DE Weitere Fehlermeldungen oder Probleme siehe:
IT Messaggi di errore o altri problemi? Fare riferimento a:
ES ¿Hay mensajes de error u otros problemas? Consulte:
NL Foutmeldingen of andere problemen? Raadpleeg:
RU Сообщения об ошибках или другие неисправности? См:
TR Hata mesajları veya başka sorunlar mı var? Bkz:
AR رسائل الخطأ أو المشكلات الأخرى؛ ارجع إلى:

13 Other Settings

1 2 3 4 5
FR Messages d'erreur ou autres problèmes? Refer to:
DE Weitere Fehlermeldungen oder Probleme siehe:
IT Messaggi di errore o altri problemi? Fare riferimento a:
ES ¿Hay mensajes de error u otros problemas? Consulte:
NL Foutmeldingen of andere problemen? Raadpleeg:
RU Сообщения об ошибках или другие неисправности? См:
TR Hata mesajları veya başka sorunlar mı var? Bkz:
AR رسائل الخطأ أو المشكلات الأخرى؛ ارجع إلى:

14 Optional Medical Accessories

FR Accessoires médicaux optionnels
DE Optionales medizinisches Zubehör
IT Accessori medicali opzionali
ES Accesorios médicos opcionales
NL Optionele medische accessoires
RU Дополнительно принадлежности
TR Opsiyonel Tıbbi Aksesuarlar
AR الملحقات الطبية الاختيارية

FR Désactivation/activation de Bluetooth
DE Deaktivieren/Aktivieren der Bluetooth-Funktion
IT Disattivazione/attivazione della funzione Bluetooth
ES Activar o desactivar el Bluetooth
NL Bluetooth uitschakelen/inschakelen
RU Выключение/включение Bluetooth
TR Bluetooth'u Devre Dışı Brakma/Etkinleştirme
AR تعطيل/تفعيل خاصية Bluetooth
FR Bluetooth est activé par défaut.
DE Die Bluetooth-Funktion ist standardmäßig aktiviert.
IT La funzione Bluetooth è attiva per impostazione predefinita.
ES El Bluetooth está activado por defecto.
NL Bluetooth is standaard ingeschakeld.
RU Bluetooth включен по умолчанию.
TR Bluetooth varsayılan olarak etkindir.
AR يتم تمكين خاصية Bluetooth بشكل افتراضي.

13.2 Restoring to the Default Settings

FR Réinitialisation aux réglages par défaut
DE Wiederherstellen der Standardeinstellungen
IT Ripristino delle impostazioni predefinite
ES Restablecimiento a los ajustes de fábrica
NL De standaardinstellingen herstellen
RU Восстановление настроек по умолчанию
TR Varsayılan Ayarları Geri Yükleme
AR استعادة الإعدادات الافتراضية

1 2 3 4 5
FR Messages d'erreur ou autres problèmes? Refer to:
DE Weitere Fehlermeldungen oder Probleme siehe:
IT Messaggi di errore o altri problemi? Fare riferimento a:
ES ¿Hay mensajes de error u otros problemas? Consulte:
NL Foutmeldingen of andere problemen? Raadpleeg:
RU Сообщения об ошибках или другие неисправности? См:
TR Hata mesajları veya başka sorunlar mı var? Bkz:
AR رسائل الخطأ أو المشكلات الأخرى؛ ارجع إلى:

1 2 3 4 5
FR Messages d'erreur ou autres problèmes? Refer to:
DE Weitere Fehlermeldungen oder Probleme siehe:
IT Messaggi di errore o altri problemi? Fare riferimento a:
ES ¿Hay mensajes de error u otros problemas? Consulte:
NL Foutmeldingen of andere problemen? Raadpleeg:
RU Сообщения об ошибках или другие неисправности? См:
TR Hata mesajları veya başka sorunlar mı var? Bkz:
AR رسائل الخطأ أو المشكلات الأخرى؛ ارجع إلى: