



OMRON®

HJ-320-E Step counter Walking style One 2.0


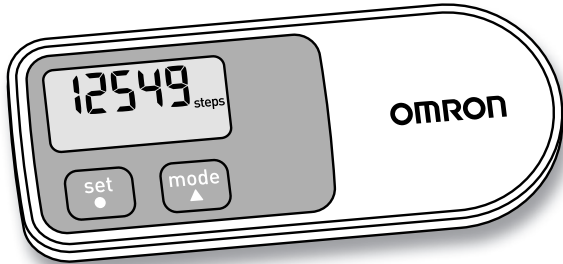


IM-HJ-320-E-03-10/2018
5331675-6C

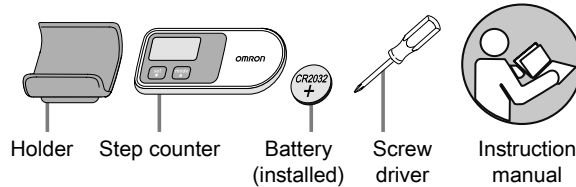

1.

- EN Attach product
- IT Fissaggio del prodotto
- DE Produkt anbringen
- FR Fixation de l'appareil
- NL Product bevestigen
- ES Colocación del producto
- RU Крепление изделия
- AR تثبيت المنتج
- BG Прикрепяне на продукта
- CS Připnutí přístroje
- DA Vedhæft produkt
- EL Προσάρτηση προϊόντος
- ET Seadme kinnitamine
- FI Mittarin kiinnitys
- HR Pričvrstite proizvod
- HU A termék rögzítése
- LT Pritvirtinkite prietaisą
- LV Ierīces piestiprināšana
- NO Feste produktet
- PL Przepięcie urządzenia
- PT Fixar produto
- RO Atașarea produsului
- SL Pripni izdelek
- SK Pripnutie zariadenia
- SV Sätta fast enheten
- TR Ürün ekle

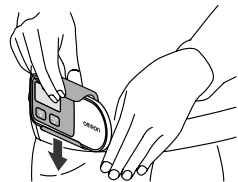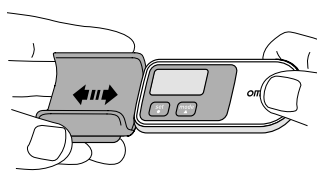


Other Languages see 8.

About the battery energy saving function

In order to save battery energy, the display of the unit will be turned off if no buttons are pressed for more than 5 minutes. However, the unit will continue to monitor your step count. Press ● or ▲ button to turn the display on again.


2.

- EN Activation: Press any key for 2 sec.
- IT Attivazione: premere un tasto qualsiasi per 2 sec.
- DE Aktivieren: Drücken Sie 2 Sekunden lang eine beliebige Taste.
- FR Activation : Appuyez sur n'importe quelle touche pendant 2 s.
- NL Inschakelen: druk 2 seconden lang op een willekeurige toets.
- ES Activar: pulse cualquier tecla durante 2 seg.
- RU Активация: нажимайте любую клавишу в течение 2 секунд.
- AR التنشيط: اضغط أي مفتاح لمدة ثانيتين.
- BG Активиране: Натиснете някой от клавишите за 2 сек.
- CS Aktivace: Stiskněte libovolné tlačítko na 2 sekundy.
- DA Aktivering: Tryk på en vilkårlig tast i 2 sek.
- EL Ενεργοποίηση: Πατήστε οποιοδήποτε πλήκτρο για 2 δευτ.
- ET Käivitamine: vajutage kahe sekundit vältel suvalist nuppu.
- FI Aktivointi: paina mitä tahansa näppäintä 2 sekuntia.
- HR Aktivacija: pritisnite bilo koju tipku na 2 sekunde.
- HU Aktiválás: Nyomja meg bármelyik gombot 2 másodpercig.
- LT Aktyvinimas: spauskite 2 sek. bet kurį mygtuką.
- LV Aktivizēšana: nospiediet jebkuru taustiņu un turiet to nospiestu 2 sekundes.
- NO Aktivering: Trykk på en hvilken som helst tast i 2 sekunder.
- PL Aktywacja: Naciskaj dowolny przycisk przez 2 sekundy.
- PT Activação: Premir qualquer tecla durante 2 segundos.
- RO Activare: Apăsati orice tastă timp de 2 secunde.
- SL Aktiviranje: Pritisnite katerokoli tipko za 2 sekundi.
- SK Aktivácia: Potlačte ľubovoľné tlačidlo v dĺžke 2 sekúnd.
- SV Aktivering: Tryck på valfri knapp i 2 sek.
- TR Etkinleştirme: 2 sn. boyunca herhangi bir tuşa basın.

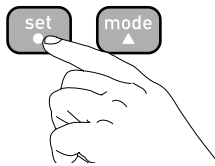

3.


- EN General Settings: Press ● for 2 sec.. Convert unit with ▲. Confirm with ●.
- IT Impostazioni generali: Premere ● per 2 sec. Convertire lo strumento con ▲. Confermare con ●.
- DE Grundeinstellung: Drücken Sie 2 Sek. die Taste ●. Wechseln Sie die Einheiten mit ▲. Bestätigen mit Taste ●.
- FR Réglages généraux : Appuyez sur ● durant 2 sec.. Convertissez les unités au moyen des touches ▲. Confirmez en appuyant sur ●.
- NL Algemene instellingen: Druk gedurende 2 s op ●. Pas de unit aan met ▲. Bevestig met ●.
- ES Ajustes generales: Pulse ● durante 2 seg. Convierta la unidad con ▲. Confirme con ●.
- RU Общие настройки: Нажмите ● и удерживайте в течение 2 с. Выберите единицу измерения с помощью кнопок ▲. Подтвердите выбор, нажав ●.
- AR إعدادات عامة: أولاً اضغط ● لمدة ثانيتين، تحويل الوحدة بـ ▲ التأكيد بـ ● ثانياً وضع البيانات
- BG Общи настройки: Натиснете ● за 2 сек.. Конвертирайте мерната единица с ▲. Потвърдете с ●.
- CS Obecné nastavení: Stiskněte ● po dobu 2 sekund. Jednotku převedte pomocí ▲. Potvrďte pomocí ●.
- DA Generelle indstillinger: Tryk på ● i 2 sek. Skift enhed med ▲. Bekræft med ●.
- EL Γενικές ρυθμίσεις: Πατήστε ● για 2 δευτ. Αλλάξτε τη μονάδα μέτρησης με τα κουμπιά ▲. Επιβεβαιώστε με το κουμπί ●.
- ET Üldsätet: Vajutage 2 sekundit nuppu ●. Teisendamiseks kasutage nuppe ▲. Kinnitamiseks vajutage nuppu ●.
- FI Yleiset asetukset: Paina ● 2 s. ajan. Muunna yksikkö painamalla ▲. Valvasta painamalla ●.

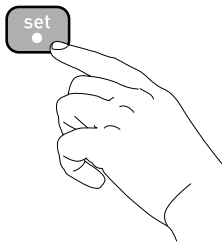

feet/inch ↔ cm

km ↔ mile


12h ↔ 24h

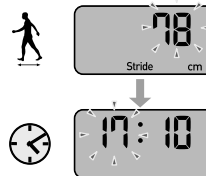

About stride length

- Measure your stride length first for accurate measurement.
- To get the most accurate results, take 10 steps and then measure the stride length for one step (from toe to toe or from heel to heel).
- The distance display is calculated by multiplying the adjusted stride length by the number of steps.


Other Languages see 8.

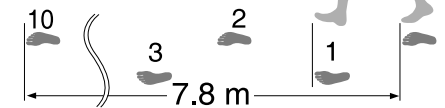

Note: If the initial settings are not correct, the measurement results may not be accurate.

4.

- EN Checking results
- IT Controllo dei risultati
- DE Ergebnisse prüfen
- FR Contrôle des résultats
- NL Resultaten controleren
- ES Comprobación de los resultados
- RU Проверка сегодняшних результатов
- AR تحقق نتائج
- BG Проверка на резултати
- CS Kontrola výsledků
- DA Kontrol af resultater
- EL Έλεγχος αποτελεσμάτων
- ET Tulemuste kontrollimine
- FI Tulosten tarkistelu
- HR Provjera rezultata
- HU Eredmények megtekintése
- LT Rezultatų tikrinimas
- LV Rezultātu apskatīšana
- NO Kontrollere resultater
- PL Sprawdzanie pomiarów
- PT Verificar os resultados
- RO Verificarea rezultatelor
- SL Preverjanje rezultatov
- SK Kontrola výsledkov
- SV Kontrollera resultat
- TR Sonuçların kontrolü

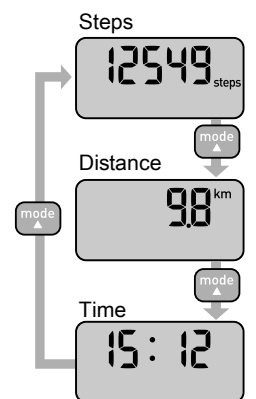

Cases where the unit may not be able to count steps accurately

Irregular movement

- When the unit is placed in a bag that moves irregularly.
- When the unit hangs from your waist or a bag.

Inconsistent walking pace

- When you shuffle or wear sandals etc.
- When you cannot walk at a consistent pace in a crowded area.

Excessive vertical movement or vibration

- When standing up and/or sitting down.
- When doing sports other than walking.
- When ascending or descending stairs or on a steep slope.
- When there is vertical or horizontal vibration in a moving vehicle such as on the bicycle, in a car, trains or a bus.


When walking very slowly

Note:

In order to avoid counting steps that are not part of a walk, the unit does not display the step count for the first 4 seconds of walking. If you continue walking for more than 4 seconds, it displays the step count for the first 4 seconds and then continues to count.


5.

- (EN) Resetting: Press ▲ for 2 sec. Confirm with ●.
- (IT) Reset: premere ▲ per 2 sec. Confermare con ●.
- (DE) Zurücksetzen: Drücken Sie ▲ 2 Sekunden lang. Bestätigen Sie mit ●.
- (FR) Réinitialisation : Appuyez sur ▲ pendant 2 s. Confirmez à l'aide de ●.
- (NL) Resetten: druk 2 seconden lang op ▲ . Bevestig met ●.
- (ES) Restaurar: pulse ▲ durante 2 seg. Confirme con ●.
- (RU) Сброс: нажимайте ▲ в течение 2 секунд. Подтвердите с помощью ●.
- (AR) إعادة الضبط: اضغط ▲ لمدة ثانيتين. قم بالتأكد باستخدام ●.
- (BG) Нулиране: Натиснете ▲ за 2 сек. Потвърдете с ●.
- (CS) Resetování: Stiskněte ▲ na 2 sekundy. Potvrďte pomocí ●.
- (DA) Nulstilling: Tryk på ▲ i 2 sek. Bekræft med ●.
- (EL) Επαναφορά ρυθμίσεων: Πατήστε το ▲ για 2 δευτ. Επιβεβαιώστε με το ●.
- (ET) Lähetestamine: vajutage kahe sekundi väitel nuppu ▲ . Kinnitage nupuga ●.
- (FI) Nollaus: Paina ▲ 2 sekuntia. Vahvista valitsemalla ●.
- (HR) Ponovno postavljanje: pritisnite ▲ na 2 sekunde. Potvrđite pomoću ●.
- (HU) Alapértékre állítás: Nyomja meg a ▲ gombot 2 másodpercig. Jóváhagyás a ● jellel.
- (LT) Kartotinis nustatymas: spauskite ▲ 2 sek. Patvirtinkite su ●.
- (LV) Atiestatīšana: nospiediet un 2 sekundes turiet nospiestu ▲ . Apstipriniet ar ●.
- (NO) Tilbakestille: Trykk på ▲ i 2 sekunder. Bekreft med ●.
- (PL) Resetowanie: Naciskaj ▲ przez 2 sekundy. Potwierdź, naciskając ●.
- (PT) Repor: Premir ▲ durante 2 segundos. Confirmar com ●.
- (RO) Resetare: Apăsăți ▲ timp de 2 secunde. Confirmați cu ●.
- (SL) Ponastavitev: Pritisnite gumb ▲ za 2 sekundi. Potrđite z gumbom ●.
- (SK) Znulovanie: Potlačte ▲ v dĺžke 2 sekúnd. Potvrďte ●.
- (SV) Återställ: Tryck på ▲ i 2 sek. Bekräfta med ●.
- (TR) Resetleme: 2 sn. boyunca ▲ tuşuna basın. ● tuşu ile onaylayın.

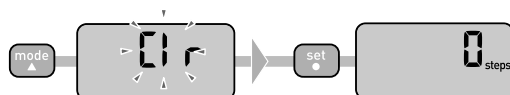

6.

- (EN) Change battery
- (IT) Sostituzione della batteria
- (DE) Batterie wechseln
- (FR) Remplacement de la pile
- (NL) Batterij vervangen
- (ES) Reemplazar pila
- (RU) Замена батареи
- (AR) تغيير البطارية
- (BG) Смяна на батерията
- (CS) Výměna baterie
- (DA) Skift batteri
- (EL) Αλλαγή μπαταρίας
- (ET) Patarei vahetamine
- (FI) Pariston vaihto
- (HR) Promjena baterije
- (HU) Elemcsere
- (LT) Elemento keitimas
- (LV) Baterijas nomaiņa
- (NO) Bytte batteri
- (PL) Wymiana baterii
- (PT) Troca de pilha
- (RO) Schimbarea bateriei
- (SL) Zamenjaj baterijo
- (SK) Výmena batérie
- (SV) Byta batteri
- (TR) Pili değiştir

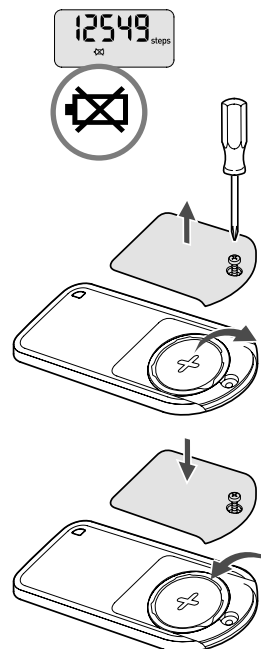

7.

Notes of safety

Symbols and definitions are as follows:

Caution: Improper use may result in injury or property damage.

Caution:

- Contact your doctor or healthcare provider before beginning a weight reduction or exercise program.
- Keep the unit out of the reach of young children.
- Always take a rest if you feel tired or experience pain while exercising. If the pain or discomfort continues, immediately stop your exercise and consult a doctor.
- If young children swallow any small piece, immediately consult a doctor.
- Do not swing the unit by the strap. Swinging the unit while holding the strap may cause an injury.
- Do not put the unit in the back pocket of your shorts or trousers. It could be damaged if you sit down while the unit is still in your pocket.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.
- If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.
- Do not insert the battery with the polarities in the wrong direction.
- Always use the type of battery indicated.
- Replace a worn battery with a new one immediately.
- Remove the battery from this unit when you are not going to use it for a long period of time (approximately 3 months or more).
- Do not throw the battery into fire because it may explode.
- Do not immerse the unit or any of the components in water. The unit is not water-proof.

OMRON HEALTHCARE EUROPE B.V. guarantees this product for 2 years after date of purchase. The guarantee does not cover battery, packaging and/or damages of any kind due to misuse (such as dropping or physical misuse) caused by the user. Claimed products will only be replaced when returned together with the original invoice / cash ticket.

Do not disassemble or modify the unit. Changes or modifications not approved by OMRON HEALTHCARE will void the user warranty.

8.

- (EN) Download the complete instruction manual at:
- (IT) Scarica il manuale d'istruzioni completo dal sito:
- (DE) Laden Sie das vollständige Handbuch mit Anweisungen von herunter.
- (FR) Téléchargez le mode d'emploi complet sur le site
- (NL) Download de volledige instructiehandleiding op:
- (ES) Descárguese el manual de instrucciones completo en:
- (RU) Загрузите полную версию руководства по эксплуатации по адресу:
- (AR) يمكنك تنزيل دليل الإرشادات بالكامل من الموقع:
- (BG) Изгледете пълните инструкции за употреба на:
- (CS) Kompletní návod k použití lze stáhnout z webu
- (DA) Hent den komplette brugsanvisning på:
- (EL) Πραγματοποιήστε λήψη του πλήρους εγχειριδίου οδηγιών στη διεύθυνση:
- (ET) Laadige täielik kasutusjuhend alla veebilehelt
- (FI) Lataa koko ohjekirja osoitteesta:
- (HR) Cjelokupni korisnički priručnik preuzmite na adresi:
- (HU) Töltse le a teljes használati útmutatót az alábbi helyről:
- (LT) Visą instrukcijų vadovą galite parsisiųsti iš interneto svetainės
- (LV) Lejupielādējiet pilnīgu lietošanas instrukciju vietnē
- (NO) Hele bruksanvisningen kan lastes ned på:
- (PL) Pełna instrukcja obsługi jest dostępna do pobrania pod adresem
- (PT) Transfira o manual de instruções completo em:
- (RO) Descărcați întregul manual de instrucțiuni de la adresa:
- (SL) Celoten priročnik z navodili za uporabo si lahko prenesete na naslovu:
- (SK) Kompletný návod na obsluhu si stiahnite zo stránky:
- (SV) Ladda ner hela instruktionshandboken på:
- (TR) Adresinden komple kullanım kılavuzunu indirebilirsiniz:

http://www.omron-healthcare.com/en/product/step_counters/HJ-320.html



Technical Data

Product name	Walking style One 2.0
Type	HJ-320-E
Power supply	3V Lithium battery type CR2032
Battery life	Approx. 6 months* (when used for 12 hours a day) Note: Supplied battery is for trial use. This battery can run out within 6 months.
Measurement range	Steps 0 to 99,999 steps Distance walked 0.0 to 5,999.9 km / 0.0 to 3,725.9 miles Time 0:00 to 23:59 / 12:00 AM to 11:59 PM
Setting range	Time: 0:00 to 23:59 / 12:00 AM to 11:59 PM Stride length: 30 to 120 cm in units of 1 cm / 12" to 48" in units of 1 inch
Operating temperature/humidity	-10°C to +40°C (14°F to 140°F)/30 to 85% RH
External dimensions	75.0(W) x 31.0(H) x 8.0(D) mm
Weight	Approx. 20 g (including battery)
Contents	Step counter, 3V lithium battery type CR2032, holder, screwdriver and instruction manual

These specifications are subject to change without notice. This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd. Japan.

* New battery life based on OMRON testing.

CE The CE-mark refers to compliance with the EMC EU-Directive 2004/108/EC

Other Languages see 8.

Battery disposal EC Directive 2008/12/EC

Batteries are not a part of your regular household waste. You must return batteries to your municipality's public collection or wherever batteries of the respective type are being sold.

Disposal of electric and electronic devices EC Directive 2002/96/EC

This product is not to be treated as regular household waste but must be returned to a collection point for recycling electric and electronic devices. Further information is available from your municipality, your municipality's waste disposal services, or the retailer where you purchased your product.

Manufacturer OMRON HEALTHCARE Co., Ltd.
53, Kunotsubo, Terado-cho, Muko, KYOTO,
617-0002 JAPAN

EU-representative OMRON HEALTHCARE EUROPE B.V.
Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS
www.omron-healthcare.com

Production facility OMRON DALIAN Co., Ltd.
No. 3, Song Jiang Road,
Economic and Technical Development Zone,
Dalian 116600, China

Subsidiaries OMRON HEALTHCARE UK LTD.
Opal Drive, Fox Milne, Milton Keynes,
MK15 0DG, UK
www.omron-healthcare.com

OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH
Konrad-Zuse-Ring 28, 68163 Mannheim, GERMANY
www.omron-healthcare.com

OMRON SANTÉ FRANCE SAS
14, rue de Lisbonne, 93561 Rosny-sous-Bois Cedex, FRANCE
www.omron-healthcare.com

Made in China