# OMRON

## Automatic Upper Arm Blood Pressure Monitor


## M2+ (HEM-7188-LE)

## M2 Essential (HEM-7188-E)

### English

## Instruction Manual 1: Safety and Other Information

For symbols information, refer to “Symbols Description” section.

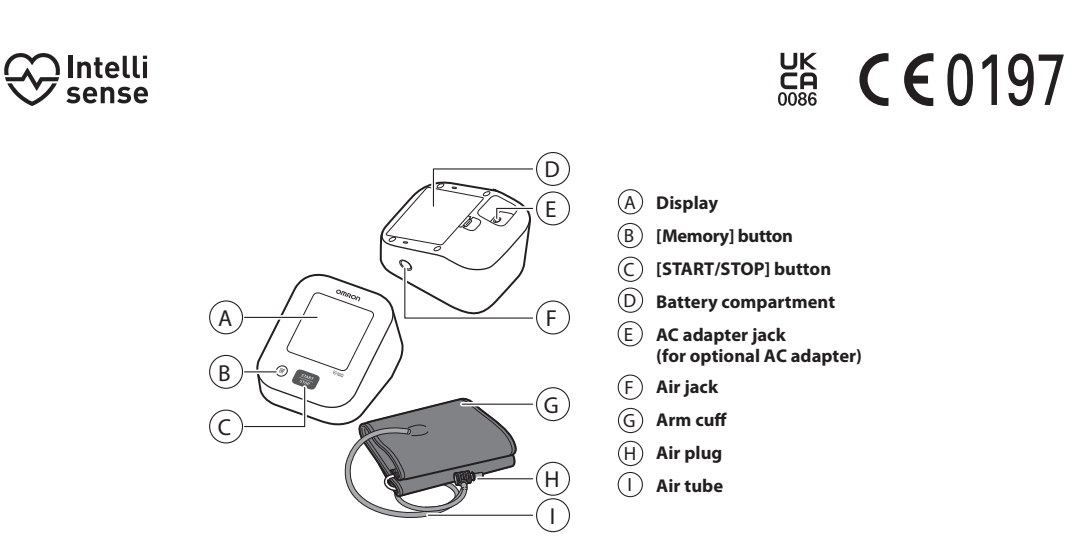- A** Display
- B** (Memory) button
- C** (START/STOP) button
- D** Battery compartment
- E** AC adapter jack (for optional AC adapter)
- F** Air jack
- G** Arm cuff
- H** Air plug
- I** Air tube

This instruction manual provides you with important information about the OMRON Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all these instructions. **If you do not understand these instructions or have any questions, contact your OMRON retail outlet or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.**

**ONLY** use the arm cuff on persons whose arm circumference is within the specified range of the cuff. Ensure that this monitor has acclimated to room temperature before taking a measurement. Taking a measurement after an extreme temperature change could lead to an inaccurate reading. If recommended that you wait for approximately 2 hours for the monitor to warm up or cool down when the monitor is used in an environment within the temperature specified as operating conditions after it is stored either at the maximum or at the minimum storage temperature. For additional information on operating and storage / transport temperature, refer to section 4. When handling the AC adapter cable: DO NOT touch the power and unplug the AC adapter immediately. DO NOT create the arm cuff or the air tube excessively. DO NOT fold or kink the air tube while taking a measurement. This may cause an injury by interrupting blood flow. To unplug the air plug, pull on the plastic air plug at the base of the tube, not the tube itself.

**ONLY** use the AC adapter, arm cuff, batteries and accessories specified for this monitor. Use of unapproved AC adapters, arm cuffs and batteries may damage and/or may be hazardous to this monitor. **ONLY** use the approved arm cuff for this monitor. Use of other arm cuffs may result in incorrect readings. DO NOT twist it. DO NOT use it if it is gathered in a bundle. / DO NOT place it under heavy objects.

**ONLY** use the approved arm cuff for this monitor. Use of other arm cuffs may result in incorrect readings. DO NOT twist it. DO NOT use it if it is gathered in a bundle. / DO NOT place it under heavy objects. Wipe any dust off of the AC adapter. Unplug the AC adapter when not in use. Unplug the AC adapter before wiping this monitor.

**AC Adapter (optional accessory) Handling and Usage** Fully insert the AC adapter into the outlet. When unplugging the AC adapter from the outlet, be sure to safely pull from the AC adapter. DO NOT pull from the AC adapter cable. Read and follow the “Correct Disposal of This Product” in section 5 when disposing of the device and any used accessories or optional parts.

**Adult Patient Population** This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population. The device detects the appearance of irregular heartbeats during measurement and indicates this via a symbol with the measurement result.

**Intended Users** Adult patient population (may include patients themselves) who can understand this instruction manual.

**Clinical Benefit** Patient’s blood pressure can be measured non-invasively and simply in the home environment.

**Type of Use** This device is intended to be multiple patient multiple use.

**Limitation** Patient’s arm circumference must be 17 - 42 cm.

**Receiving and Inspection** Remove this monitor and other components from the packaging and inspect for damage. If this monitor or any other components is damaged, DO NOT USE and consult with your OMRON retail outlet or distributor.

## Important Safety Information

Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety. **For specific information about your own blood pressure, consult with your physician.**

**Contraindications** DO NOT use this monitor on an injured arm or an arm under medical treatment. DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion. DO NOT use this monitor on infants, toddlers, children or persons with severe tremors themselves.

**Side Effects** Taking measurements more often than necessary may cause bruising due to blood flow interference. Inflating to a higher pressure than necessary may result in bruising or pain. The arm cuff contains lead. NOTE: Refer to “If your systolic pressure is more than 210 mmHg” in the end of instruction manual 2 for additional information.

**Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.**

DO NOT adjust medication including altering your use of any drug or treatment based on readings from this monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure and heart related conditions. NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician. If you are experiencing any symptoms or concerns, consult with your physician.

DO NOT postpone/stop regular checkups or physician visits based on readings from this monitor. DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computed tomography (CT) scans, etc. This may result in incorrect operation of the monitor and/or cause an inaccurate reading. DO NOT use this monitor in oxygen rich environments or near flammable gas. Consult with your physician before using this monitor if you have common medical conditions such as atrial fibrillation, pressure monitors or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions may affect the accuracy of the monitor, trembling, or shivering may affect the measurement reading. To help avoid strangulation, keep the air tube and AC adapter cable away from infants, toddlers and children. DO NOT use this monitor in areas where a choking hazard if swallowed by infants, toddlers and children.

**AC Adapter (optional accessory) Handling and Usage** DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged. Do not touch the power and unplug the AC adapter immediately. Plug the AC adapter into the appropriate voltage outlet. DO NOT use a multi-outlet plug. NEVER plug in or unplug the AC adapter from the electric outlet with wet hands. Do not attempt or attempt to repair the AC adapter.

**Battery Handling and Usage** Keep the batteries out of the reach of infants, toddlers and children. **Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.**

Consult with your physician before using this monitor on an arm where intravascular access or therapy, or an antero-venous (AV) fistula is present because of temporary interference to blood flow and could result in injury. Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising. ONLY inflate the arm cuff when it is applied on your upper arm. Remove the arm cuff if it does not start deflating during a measurement. During measurement, make sure that no mobile device or any other electrical device that emits electromagnetic fields is within 30 cm of this monitor. This may result in incorrect operation of the monitor and/or cause an inaccurate reading. DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading. DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor. DO NOT use this monitor in a moving vehicle such as in a car or on an aircraft.

DO NOT drop or subject this monitor to strong shocks or vibrations. DO NOT use use this monitor in places with high or low humidity or high or low temperature. Refer to section 4. During measurement, observe the arm to ensure that the monitor is not causing prolonged impairment to blood circulation. DO NOT use this monitor with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation of the devices and/or cause an inaccurate reading.

Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for at least 10 minutes before taking a measurement. Rest for at least 5 minutes before taking a measurement. Remove tight-fitting or cloth clothing from your arm while taking a measurement. Remain still and DO NOT talk while taking a measurement.

Display/Problem	Possible Cause	Solution
appears		Apply the arm cuff correctly. Do not take another measurement. Refer to section 4 of instruction manual 2. Remain still and sit correctly during a measurement. If the irregular heartbeat symbol
appears	The pulse rate is not detected correctly.	continues to appear. It is recommended that you consult with your physician.
does not flash during a measurement		Replacing all 4 batteries with new ones is recommended. Refer to section 3 of instruction manual 2.
flashes	Batteries are low.	
and  appears, or the monitor is turned off unexpectedly during a measurement.	Batteries are depleted.	Immediately replace all 4 batteries with new ones. Refer to section 3 of instruction manual 2.
appears, or the monitor is turned off unexpectedly during a measurement.	Battery polarities are not properly aligned.	Check the battery installation for proper placement. Refer to section 3 of instruction manual 2.
Nothing appears on the display of the monitor.	Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review sections 2, 4 and 5 of instruction manual 2.	
Readings appear too high or too low.		Press the (START/STOP) button to turn the monitor off, then press it again to take a measurement. If the problem continues, remove all batteries and wait for 30 seconds. Then re-install the batteries. If the problem still persists, contact your OMRON retail outlet or distributor.
Any other problem occurs.		

Memory	Stores up to 30 readings
Contents	Monitor, arm cuff (HEM-RML31 for HEM-7188-LE / HEM-CR24 for HEM-7188-E, 4 “AA” batteries, instruction manual 1 and 2
Protection against electric shock	Internally powered ME equipment (when using only batteries) Class II ME equipment (when using optional AC adapter)
Applied part	Type BF arm cuff)
<b>Note</b>	<b>Note</b> <ul style="list-style-type: none"><li>These specifications are subject to change without notice.</li> <li>This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2019+A1:2020-A2:2024 (excluding pregnant and pre-eclampsia patients). In the clinical validation study, it was used on 85 subjects for determination of diastolic blood pressure. If classification is degrees of protection provided by enclosures in accordance with IEC 60529. This monitor and optional AC adapter are protected against solid foreign objects of 12.5 mm diameter and greater such as a finger. The monitor and optional AC adapter (HEM-CR24 and HHP-AM01) are protected against falling water drops which may cause issues during a normal operation. The optional AC adapter HHP-BFH01 is protected against oblique falling water drops which may cause issues during a normal operation.</li> <li>Operation mode classification complies with IEC 60601-1.</li></ul>

3 Correct Disposal of This Product (Waste Electrical & Electronic Equipment)	
This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.	
<b>6 Important Information regarding Electromagnetic Compatibility (EMC)</b>	
This device conforms to the EN 60601-1-2:2015+A1:2021 Electromagnetic Compatibility (EMC) standard. Further documentation in accordance with this EMC standard is available at <a href="https://healthcare.omron.com/electro-magnetic-compatibility">https://healthcare.omron.com/electro-magnetic-compatibility</a> . Refer to the EMC information for this device on the website.	

## 2 Limited Warranty

Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing. It is designed to give you a high level of comfort, provided that it is properly operated and maintained as described in the instruction manual.

ONLY use the AC adapter warranted by OMRON for a period of 5 years after the date of purchase. The proper construction, workmanship and materials of this product is warranted by OMRON. During this period of warranty OMRON will, without charge, at the end of its working life, repair or replace the defect product or any defective parts. The warranty does not cover any of the following:

- A. Transport costs and risks of affr transport
- B. Costs for repairs and / or defects resulting from repairs done by unauthorised persons.
- C. Periodic check-ups and maintenance.
- D. Failure or wear of optional parts or other attachments other than the main device itself, unless explicitly warranted above.
- E. Costs arising due to non-acceptance of a claim (those which will be charged for).
- F. Damages of any kind including personal damage accidentally or from misuse.
- G. Calibration service is not included within the warranty.
- H. Optional parts have a one (1) year warranty from date of purchase.

Optional parts include, but are not limited to the following items: cuff and cuff tube.

Should warranty service be required please apply to the dealer whom the product was purchased from or an authorised OMRON distributor. For the address refer to the product packaging / literature or to your specialised retailer. If you have difficulties in finding OMRON customer services, visit our website ([www.omron-healthcare.com](http://www.omron-healthcare.com)) for contact information. Repair or replacement under the warranty does not give rise to any extension or renewal of the warranty period.

The warranty will be granted only if the complete product is returned together with the original invoice / cash ticket issued to the consumer by the retailer.

## 3 Maintenance

**3.1 Maintenance**

- Protect your monitor and do not damage, follow the directions below. Changes or modifications not approved by the manufacturer will void the user warranty.

**⚠️ Precaution**
DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

## 3.2 Storage

- Store your monitor and other components in a clean, safe location.
- Gently fold the air tube into the arm cuff. Do not bend or crease the air tube excessively.
- Do not store your monitor and other components:
  - If your monitor and other components are wet.
  - In locations exposed to extreme temperatures, humidity, direct sunlight or corrosive vapors such as bleach.
  - In locations exposed to vibrations or shocks.

**3.3 Wiping the Monitor**

- Do not use any abrasive or volatile cleaners.
- Use a soft dry cloth or a soft cloth moistened with mild (neutral) detergent to wipe your monitor and arm cuff, and then wipe them with a dry cloth.
- Do not wash or immerse your monitor and arm cuff or other components in water.
- Do not use gasoline, thinners or similar solvents to wipe your monitor and cuff or other components.

## 3.4 Calibration and Service

- The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
- It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON contact details at the address given on the packaging or attached literature.

## 4 Specifications

Product Category	Electronic Sphygmomanometers
Product description	Automatic Upper Arm Blood Pressure Monitor
Model (Code)	M2+ (HEM-7188-LE) / M2 Essential (HEM-7188-E)
Display	LCD digital display
Blood pressure range	0 to 299 mmHg
Blood pressure measurement	SYS: 60 to 260 mmHg / DIA: 40 to 115 mmHg
Pulse measurement range	40 to 180 beats / min.
Accuracy	Pressure: ±3 mmHg / Non-com: ±5 % of display reading
Measurement method	Oscillometric method
Operation mode	Continuous operation
IP classification	Monitor: IP21 / Optional AC adapter: IP21 (HHP-AM01 / HHP-AM01) or IP22 (HHP-BFH01)
Battery life	Approx. 900 measurements (Using new alkaline batteries and included arm cuff. Depending on the type of battery and arm cuff)
Durable period (Service life)	Monitor: 5 years or the time when reaching 30000 times of use. / Cuff: 5 years or the time when reaching 10000 times of use. / Optional AC Adapter: 5 years /
Operating conditions	+10 to +40 <span> </span> °C / 15 to 90 <span> </span> % RH (non-condensing) / 1000 hPa
Storage / Transport conditions	-20 to +60 <span> </span> °C / 10 to 90 <span> </span> % RH (non-condensing)
Weight	Monitor: approx. 260 g (not including battery) / Arm cuff for HEM-7188-LE: approx. 170 g / Arm cuff for HEM-7188-E: approx. 110 g
Dimensions	Monitor: approx. 98 mm (W) x 79 mm (H) x 138 mm (L) / HEM-7188-LE: approx. 145 mm x 594 mm (air tube: 750 mm) / Arm cuff for HEM-7188-LE: approx. 466 mm x 466 mm (air tube: 610 mm)
Cuff circumference applicable to the monitor	17 to 42 cm (included arm cuff: HEM-7188-LE: 22 to 42 cm HEM-7188-E: 22 to 32 cm)

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- Relax and sit comfortably. Remain still and do not talk.
- 1. Keep your back and arm supported.
- 2. Keep the arm cuff at the same level as your heart.
- 3. Keep your feet flat and your legs uncrossed.

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- 1. Relax and sit comfortably. Remain still and do not talk.
- 2. Keep your back and arm supported.
- 3. Keep your feet flat and your legs uncrossed.

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- 1. Press the [START/STOP] button.
- A: Flashes at every heartbeat.
- B: Appears while the cuff is deflating.
- 2. The reading is saved automatically.

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