

What does Viso look like in the clinician dashboard? The Viso

platform consists of a dashboard for clinicians and an app for

patients.

Here you see the initial screen of the Viso clinician dashboard,

which is the monitoring screen. On the monitoring screen you can

view an overview of all the patients registered in the

system.

Using the search bar, you can find individual patients. You

can also filter on various factors such as programs or

assigned approvers.

To view an individual patient, simply click on a patient.

This is the patient screen. Here you can look at what actions are

available to act on as well as view the data for each patient.

You can also manage the programs the patient is assigned to as

well as manage the patient profile.

If you click on the arrow, you go back to the home screen. If

you press the hamburger icon you can expand or retract the side

menu.

In the side menu you can navigate to the other screens of

the dashboard.

The red dot with the number represents the amount of

patients that have actions available to them. If you press

the "Enrollment" button, it will take you to the enrollment

screen. In this screen, you can enroll new patients to the Viso

platform. You can also view the progress of patients that are

not yet completely enrolled.

If you press the "Programmes" button, it will take you to the

programmes screen. In the programmes screen you can view and edit the

available programmes of the Viso platform.

If you press the "Manage users" button, it will take you to the

manage users screen. In the manage users screen you can view

all the users that have access to the Viso clinician dashboard.

You can also add new users or even remove them. If you press

the "Help" button it will take you to the help screen.

On the help screen, you can access information regarding how

to perform certain tasks in Viso, as well as training videos

and other relevant information. If you press the "settings"

button, it will take you to the settings screen. On the settings

screen, you can access your personal information and change

it. Finally, you can log out using the "Logout" button.