

Reviewing patient data.

Viso is a great tool to keep up with your patient and their

health status. Let's look at some ways we can review the data

from the patients registered on Viso. When on the patient profile

of the patient you want to review, click on one of the

measurement unit boxes to go to the detailed monitoring screen.

You can see a graph of the blood pressure measurements the

patient has recorded.

To view blood pressure data from an earlier measurement point,

you can adjust the window of time by choosing an earlier date

in the drop down menu.

If you want to see all measurements in list form, press

the "View all measurements" text box.

Scroll down to see the other indices that are being

monitored.

To look at the monthly questionnaire results, press

"Questionnaire" in the top bar. Here you can see the

questionnaire that the patient answered, their current and

previous answers.

Press "View history" to look at the results of the questionnaire

from further back.

To view the results of the lab tests, click the "Lab test" button

in the top bar. Here you can find the current and previous

results of the lab tests.

To view the results of the assessments taken before an in

clinic visit, press the "Assessments" button in the top

menu.

Here again you can see the current and previous results of

the assessments taken by the patient. Finally, by pressing

the "Review" button, you can find details of the monitoring

programmes. You can find a list of outstanding reviews or trigger

new ones yourself.

Click the review you want to send out and click "Contact

patient", then leave a message for the patient. The screen also

shows the managed conditions the patient is being monitored for

and the current medication the patient is taking.

This information is retrieved from the ER, so it may differ

from the information present in the Viso environment. If there

is an incongruency, you can choose to change the patient's

medication in the Viso environment.